

NeoShades® with Tabs

PHOTOTHERAPY EYE SHIELDS



FEATURES & BENEFITS

- Soft, padded eye shield with cute sunglasses design
- Ocular pockets designed for patient comfort and safety (N724, N725, N726)
- Skin friendly NeoBond® Hydrocolloid tabs hold NeoShades in place
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT NO.	ITEM	QTY/UNIT
N720	Micro	50/box
N721	Small	50/box
N722	Large	50/box
With Ocular Pockets		
N724	Micro	50/box
N725	Small	50/box
N726	Large	50/box

U.S. Patent #8,070,288;
Additional Patents Pending



NeoShades® with Tabs

PHOTOTHERAPY EYE SHIELDS

INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Preparation

- 1 Select proper size.

Approximate sizing:
Micro – 28 weeks and under
Small – 29-36 weeks
Large – 37 weeks and up

- 2 Clean and dry skin on temples per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

DO NOT USE ALCOHOL.



Application


- 1 Carefully remove hydrocolloid tabs from eye shields.

Before removing plastic liners, hydrocolloid adhesive tabs must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.

- 2 Peel and discard plastic liners. Place hydrocolloid adhesive tabs on skin temples and hold in place for 60 seconds to ensure proper adhesion.

- 3 Cover eyes with NeoShades and gently attach to hydrocolloid adhesive tabs.


Note: Leave a portion of the tab exposed for easier removal.

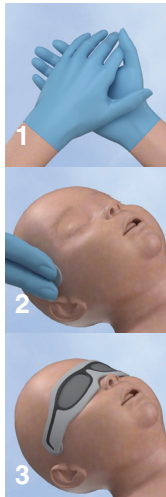

Removal

- 1 Replace NeoShades hydrocolloid adhesive tabs on skin every 5 days or per hospital protocol, whichever is sooner.
- 2 Hold edge of hydrocolloid adhesive tab in place and slowly separate NeoShades from tab.
- 3 Saturate hydrocolloid adhesive tab with water or saline.
- 4 Slowly peel hydrocolloid adhesive tab away from skin as you swab with water or saline.

Repeat for opposite side.

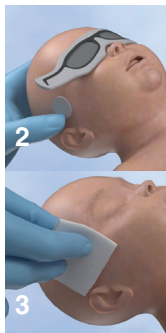

GENERAL INFORMATION

Light Blocking

99.9%

Product Materials

NeoShades: Nylon, polyurethane
Tabs: NeoBond Hydrocolloid, hook fabric
This product is not made with natural rubber latex or plasticizer DEHP.

Size

N720, N724 Micro: 108 mm
(fits head circumference < 26 cm)
N721, N725 Small: 127 mm
(fits head circumference 26-34 cm)
N722, N726 Large: 178 mm
(fits head circumference > 34 cm)

Packaging

Individually packaged: Yes
Package type: Polyethylene bag
Dispenser box: Yes (N720, N721, N722 only)

Visit neotechproducts.com for the most up-to-date information.

Boxes are printed with environmentally friendly ink on recyclable material.

CONTRAINDICATIONS, CAUTIONS & WARNINGS

The NeoShades with Tabs are contraindicated for patients with facial deformation, skin infection or excoriated skin areas that would prevent the proper placement of the tabs. They are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Tabs should always be removed when NeoShades are taken off the patient. Do not reuse.

When using NeoShades intermittently, always assess the integrity of the hydrocolloid tabs whenever the eye shields are removed. If the hydrocolloid tabs are pulling away from the skin, remove per the directions for use and apply new hydrocolloid tabs before resuming phototherapy.

Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Discontinue use immediately if skin irritation occurs.

Single patient use only.

General transportation and storage practices are applicable.

Certified



Corporation



Made in USA

For more information or to order your FREE sample call

1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355



EMERGO EUROPE
Westervoortsedijk 60
6827 AT, Arnhem
The Netherlands



PPE Regulation (EU) 2016/425
Notified Body: BSI Group (2797),
Say Building, John M. Keynesplein 9
1066 EP Amsterdam, Netherlands