


NeoShades® with Headstrap

PHOTOTHERAPY EYE SHIELDS

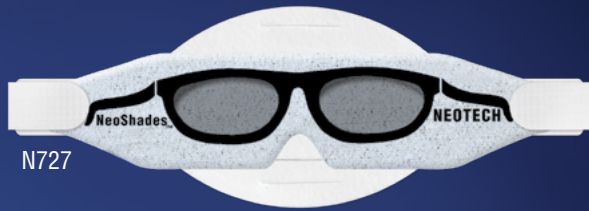

FEATURES & BENEFITS

- Soft, padded eye shields with cute sunglasses design
- Ocular pockets designed for patient comfort and safety
- Soft, patented NeoFoam® headstrap holds NeoShades in place
- Adjustable headstrap engineered to minimize slipping and help reduce head molding
- Unique headstrap shape designed to ensure ideal fit
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT. #	ITEM	QTY/UNIT
N727	Micro	50/box
N728	Small	50/box
N729	Large	50/box

U.S. Patent #8,070,288;
Additional Patents Pending



NeoShades® with Headstrap

PHOTOTHERAPY EYE SHIELDS

INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

1 Select proper size.

Approximate sizing:

Micro – 28 weeks and under

Small – 29-36 weeks

Large – 37 weeks and up


2 Place headstrap behind head with smooth side of foam against head.

3 Cover eyes with NeoShades.

4 Gently attach hook fabric on end of headstrap to NeoShades.

Repeat for opposite side.

Note: Straps can be trimmed for proper fit.

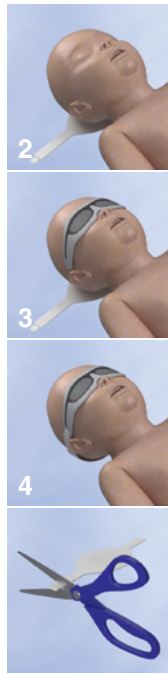

GENERAL INFORMATION

Light Blocking

99.9%

Product Materials

NeoShades: Nylon, polyurethane

Headstrap: NeoFoam, hook fabric

This product is not made with natural rubber latex or plasticizer DEHP.

Size

N727 Micro: 108 mm

(fits head circumference < 26 cm)

N728 Small: 127 mm

(fits head circumference 26-34 cm)

N729 Large: 178 mm

(fits head circumference > 34 cm)

Packaging

Individually packaged: Yes

Package type: Polyethylene bag

Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

Boxes are printed with environmentally friendly ink on recyclable material.

CAUTIONS & WARNINGS

The NeoShades are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Single patient use only.



For more information or to order your **FREE** sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355



EMERGO EUROPE
Westervoortsedijk 60
6827 AT, Arnhem
The Netherlands



PPE Regulation (EU) 2016/425
Notified Body: **BSI Group (2797)**,
Say Building, John M. Keynesplein 9
1066 EP Amsterdam, Netherlands