

## NeoShades<sup>®</sup> with Headstrap

### PHOTOTHERAPY EYE SHIELDS



#### FEATURES & BENEFITS

- Soft, padded eye shields with cute sunglasses design
- Ocular pockets designed for patient comfort and safety
- Soft, patented NeoFoam<sup>®</sup> headstrap holds NeoShades in place
- Adjustable headstrap engineered to minimize slipping and help reduce head molding
- Unique headstrap shape designed to ensure ideal fit
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT. #	ITEM	QTY/UNIT
N727	Micro	50/box
N728	Small	50/box
N729	Large	50/box

U.S. Patent #8,070,288;  
Additional Patents Pending



# NeoShades® with Headstrap

## PHOTOTHERAPY EYE SHIELDS

### INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

- 1 Select proper size.

Approximate sizing:

Micro – 28 weeks and under

Small – 29-36 weeks

Large – 37 weeks and up


- 2 Place headstrap behind head with smooth side of foam against head.

- 3 Cover eyes with NeoShades.

- 4 Gently attach hook fabric on end of headstrap to NeoShades.

Repeat for opposite side.

*Note: Straps can be trimmed for proper fit.*

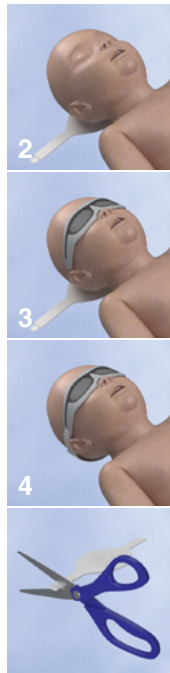

### GENERAL INFORMATION

#### Light Blocking

99.9%

#### Product Materials

NeoShades: Nylon, polyurethane

Headstrap: NeoFoam, hook fabric

This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

N727 Micro: 108 mm

(fits head circumference < 26 cm)

N728 Small: 127 mm

(fits head circumference 26-34 cm)

N729 Large: 178 mm

(fits head circumference > 34 cm)

#### Packaging

Individually packaged: Yes

Package type: Polyethylene bag

Dispenser box: Yes

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CAUTIONS & WARNINGS

The NeoShades are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Single patient use only.



For more information or to order your **FREE** sample call  
**1-800-966-0500** or visit **[neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355



EMERGO EUROPE  
Westervoortsedijk 60  
6827 AT, Arnhem  
The Netherlands



PPE Regulation (EU) 2016/425  
Notified Body: **BSI Group (2797)**,  
Say Building, John M. Keynesplein 9  
1066 EP Amsterdam, Netherlands