

Telemetry Totes



FEATURES & BENEFITS

Neotech Clip-Tote

- Our most popular pouch
- Water resistant
- Secure clip for easy attachment
- Allows for patient mobility

Neotech Valu-Tote

- Our most economical pouch
- Secure clip for easy attachment
- Allows for patient mobility

CAT. #	Product	Dimensions	QTY/UNIT
A501	Clip-Tote	5" x 8"	100/box
A507	Valu-Tote	4.75" x 8.75"	100/box



Telemetry Totes

DIRECTIONS FOR USE

- 1 Open tote at zipper.
- 2 Insert telemetry transmitter into tote.
- 3 Thread lead wires through slit and connect to transmitter.
- 4 Seal zipper.
- 5 Secure clip to patient gown or robe.



For more information or to order your **FREE** sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2022 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



M524 Rev E