


The original USA made

## Little Sucker<sup>®</sup> Nasal Tip & Cover

NASAL SUCTION DEVICE AND PROTECTIVE COVER

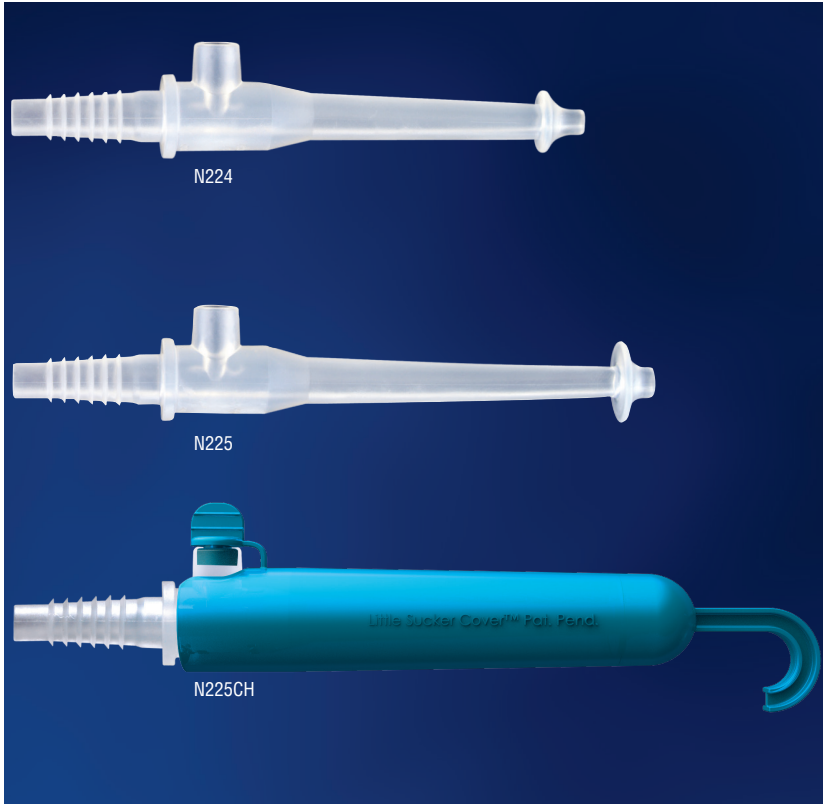

### FEATURES & BENEFITS

#### Little Sucker Nasal Tip

- Specialized tip designed to minimize potential tissue trauma
- Designed for single handed suctioning
- Thumb port for intermittent suctioning
- Soft, flexible tip similar to a bulb syringe
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

#### Little Sucker Cover

- Designed to protect the Little Sucker from damage, debris and external contaminants
- Convenient hook for hanging (N217, N225CH)
- Bright blue color for better visibility
- Noise reducing port plug
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	ITEM	QTY/UNIT
N224	Preemie Nasal Tip	50/box
N224TP	Preemie 2-piece	50/box
N225	Standard Nasal Tip	50/box
N225TP	Standard 2-piece	50/box
N225CH	Standard Nasal w/Hook Cover	50/box
N217	Cover with Hook for N225	50/box*

\*Not compatible with N224, N224TP, N225TP

U.S. Patent  
 #7,635,361; # D668,757;  
 #6,958,050; #D474,269



# Little Sucker® Nasal Tip & Cover

## NASAL SUCTION DEVICE AND PROTECTIVE COVER

### INDICATIONS FOR USE

The Little Sucker Nasal Tip is indicated for use as a tip for suction tubing to aspirate fluids from the nares.

### DIRECTIONS FOR USE

- 1 Connect barbed end of Little Sucker Nasal Tip to suction line.
- 2 Set suction to appropriate pressure per hospital protocol or physician recommendation.
- 3 Block thumb port with thumb to begin suctioning.
- 4 Suction nares with intermittent suction. Remove thumb from thumb port to stop suctioning. Block thumb port again to continue suctioning.
- 5 Suction sterile water through Little Sucker Nasal Tip after each use to clear debris.
- 6 Replace Little Sucker Nasal Tip every 24 hours or per hospital protocol, whichever is sooner.

### GENERAL INFORMATION

#### Product Materials

Little Sucker: TPE (thermoplastic elastomer)  
This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

N224  
Length: 106 mm  
Tip opening diameter: 2 mm  
Nasal guard diameter: 11 mm  
Thumb port opening diameter: 5 mm

N224TP  
Length: 109 mm  
Tip opening diameter: 2 mm  
Nasal guard diameter: 11 mm  
Thumb port opening diameter: 6 mm

N225  
Length: 120 mm  
Tip opening diameter: 3 mm  
Nasal guard diameter: 15 mm  
Thumb port opening diameter: 5 mm

N225CH  
Length: 152.5 mm  
Tip opening diameter: 3 mm  
Nasal guard diameter: 15 mm  
Thumb port opening diameter: 5 mm

N225TP  
Length: 109 mm  
Tip opening diameter: 2 mm  
Nasal guard diameter: 15 mm  
Thumb port opening diameter: 6 mm

N217  
Length: 121.4 mm

#### Packaging

Individually packaged (N225CH: Sucker and Cover packaged together)

Package type: Polyethylene bag

Dispenser box

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

For nasal use only.

Inserting the Little Sucker Nasal Tip too far into the nares can cause tissue trauma.

The Little Sucker Cover is designed for use with Little Sucker devices specifically. It may not accommodate other suction devices. It is not intended for use with the Little Sucker 2-piece devices.

The Little Sucker Nasal Tip should remain free of debris.

Single patient use only.



For more information or to order your **FREE** sample call  
**1-800-966-0500** or visit **[neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2024 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



Made in USA

M534 Rev J