


## Neotech ChinStrap<sup>™</sup>

### RESPIRATORY SUPPORT ACCESSORY

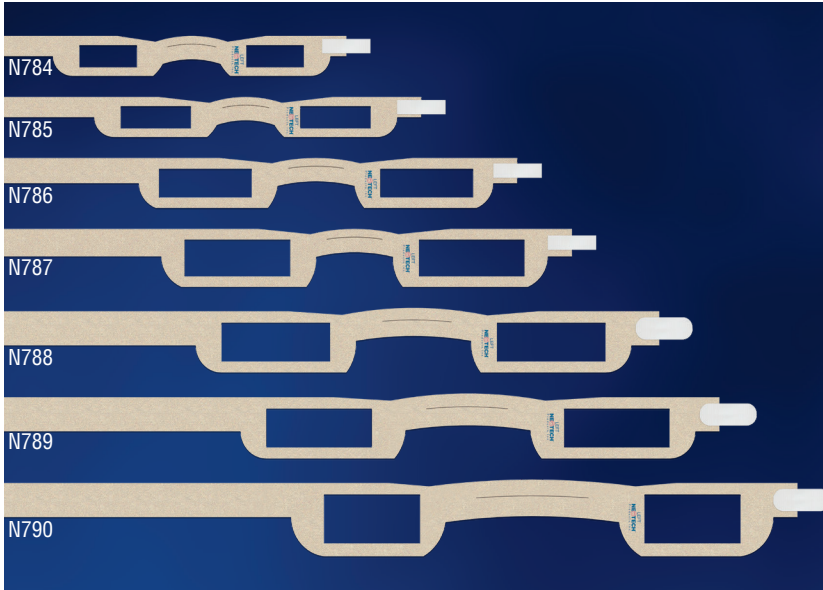

#### FEATURES & BENEFITS

- Soft, breathable, skin friendly NeoFoam<sup>®</sup> wicks perspiration
- Flexibility enables minimal mouth movement
- Form fitting and adjustable
- Split chin design for comfort and stability
- Ear openings to prevent folding over
- Designed to reduce the risk of pressure ulcers or necrosis
- Unique material helps to prevent sliding
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Item	Qty/Unit
N784	Nano	20/box
N785	Micro	20/box
N786	Small	20/box
N787	Large	20/box
N788	X-Large	20/box
N789	Macro	20/box
N790	Jumbo	20/box

U.S. Patent #10,342,693



# Neotech ChinStrap™

## RESPIRATORY SUPPORT ACCESSORY

### INDICATIONS FOR USE

The Neotech ChinStrap is intended to help keep a patient's mouth closed during respiratory therapy. It is intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

- 1 Select the proper size by measuring from the center of the chin, along the ear, across the crown of the head, along the other ear, and back to the center of the chin.

N784 Nano: fits head circumference 22-27 cm

N785 Micro: fits head circumference 28-35 cm

N786 Small: fits head circumference 33-42 cm

N787 Large: fits head circumference 36-46 cm

N788 X-Large: fits head circumference 38-53 cm

N789 Macro: fits head circumference 39-58 cm

N790 Jumbo: fits head circumference 40-66 cm



- 2 Position the Neotech ChinStrap on the chin with the thinner portion across the front of the chin and the wider portion under the chin.

*Note: Only the white side of the strap should be placed against the skin*

- 3 Gently pull strap toward posterior crown of head with ears positioned inside ear openings and then fasten hook tab to strap. Side marked "LEFT" should be under patient's left ear.

*Note: May be trimmed for proper fit. Ensure that strap is not too tight or pressing on ears. Hook tab should completely overlap strap.*

- 4 Replace every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if worn or visibly soiled.



### GENERAL INFORMATION

#### Product Materials

NeoFoam, hook fabric

This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

N784 Nano: fits head circumference 22-27 cm

N785 Micro: fits head circumference 28-35 cm

N786 Small: fits head circumference 33-42 cm

N787 Large: fits head circumference 36-46 cm

N788 X-Large: fits head circumference 38-53 cm

N789 Macro: fits head circumference 39-58 cm

N790 Jumbo: fits head circumference 40-66 cm

#### Packaging

Individually packaged: Yes

Package type: Polyethylene bag

Dispenser box: Yes (N784-N787)

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CAUTIONS & WARNINGS

The Neotech ChinStrap is not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

Neotech ChinStrap should only be used while patient is under continuous direct supervision of healthcare professionals. ChinStrap can pose a potential strangulation hazard.

Discontinue immediately if skin irritation occurs.

Single patient use only.



**For more information or to order your FREE sample call  
1-800-966-0500 or visit [neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2022 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



M551 Rev G