


NeoGrip[®]

TUBING AND CABLE HOLDER

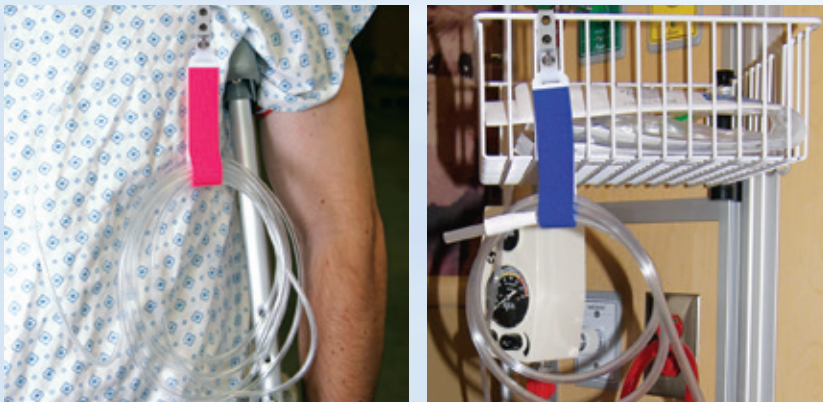

FEATURES & BENEFITS

- Helps to keep tubes, cables and circuits together
- Allows for better patient mobility
- Attaches to linen or clothing without damaging
- Helpful during kangaroo care
- Open and close repeatedly with no residue
- Can be used in clinical or home care setting
- Great for ambulatory patients
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	COLOR	QTY/UNIT
N601	Teal (VersaGrip)	40/box
N602	Pink/Blue	40/box
N602B	Blue	40/box
N602P	Pink	40/box
N602R	Red	40/box
N602G	Green	40/box
N602Y	Yellow	40/box
N602O	Orange	40/box
N602V	Violet	40/box
N603	White	40/Box

Patent #D814,568



Clinician Invented



Made in USA

NeoGrip®

TUBING AND CABLE HOLDER

INDICATIONS FOR USE

The NeoGrip/VersaGrip is intended to organize tubes, cables and circuits.

DIRECTIONS FOR USE

- 1 Separate hook and loop material.
- 2 Insert tubes, cables and circuits.
- 3 Align hook and loop and apply pressure to secure.
- 4 To remove, slowly peel hook and loop apart and carefully remove tubes, cables, and circuits.

Note: Replace immediately if hook and loop is worn.

GENERAL INFORMATION

Product Materials

Hook and loop fabric, mylar, nickel plated steel
This product is not made with natural rubber latex or plasticizer DEHP.

Size

NeoGrip

Closed length: 115 mm (not including clip)
Width: 25 mm

VersaGrip (N601)

Closed length: 150 mm (not including clip)
Width: 25 mm

Packaging

Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

CAUTIONS

Do not use if hook or loop fabric is worn.

Single patient use only.



For more information or to order your **FREE** sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355



Made in USA