


# Neotech Meconium Aspirator<sup>®</sup>

## MECONIUM REMOVAL DEVICE

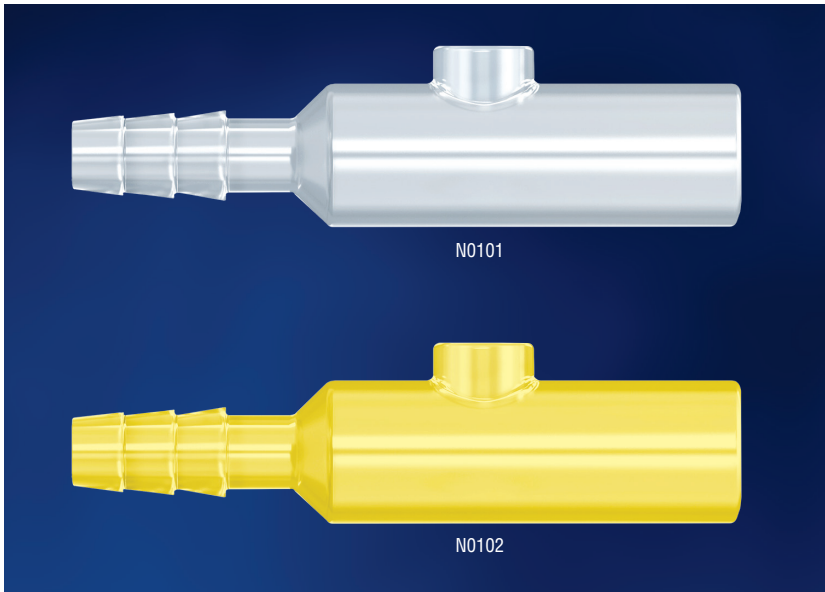


### FEATURES & BENEFITS

- Designed to eliminate mouth suctioning of meconium
- Suction control port for intermittent suctioning
- Attaches to most endotracheal tubes
- Featured in the Textbook of Neonatal Resuscitation<sup>1</sup>
- N0102 (yellow) allows for easier identification
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

1) Textbook of Neonatal Resuscitation, 6th Edition (2011) pg. 43, Figure 2.7

CAT. #	ITEM	QTY/UNIT
N0101	Clear	40/box
N0102	Yellow	40/box

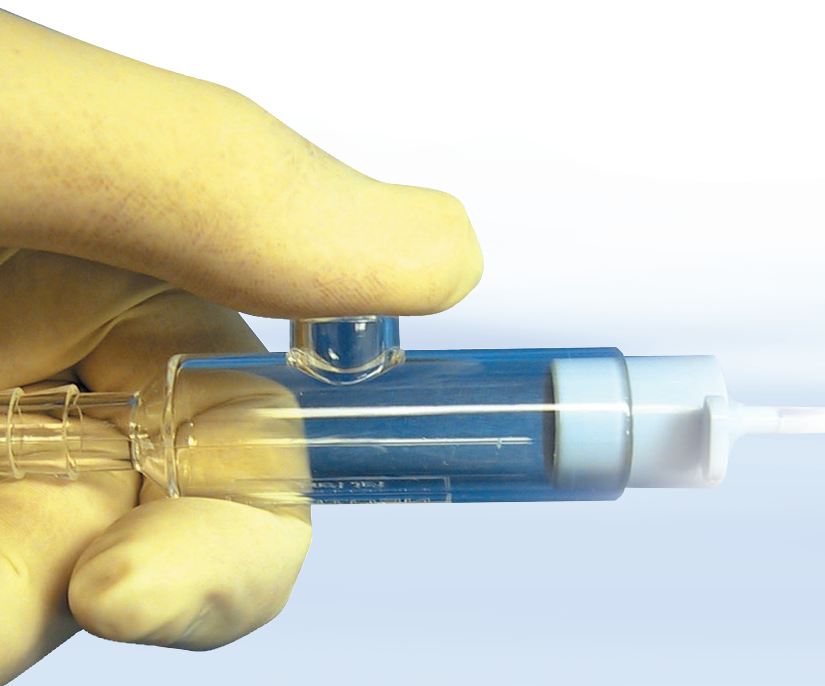

Made in USA

# Neotech Meconium Aspirator®

## MECONIUM REMOVAL DEVICE

### INDICATIONS FOR USE

The Neotech Meconium Aspirator is intended for use as a suction device to aspirate meconium from a newborn's upper airway.

### DIRECTIONS FOR USE

- 1 Connect barbed end of Neotech Meconium Aspirator to suction line.
- 2 Set the suction pressure at 80mm Hg or less.
- 3 Intubate patient with proper ET tube.
- 4 Connect 15mm I.D. end of Neotech Meconium Aspirator to 15mm O.D. ET tube adapter.
- 5 Block thumb port with thumb to begin suctioning.
- 6 Suction for two seconds or less at a time until meconium is full removed. Suction meconium intermittently.  
*Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.*
- 7 Continue to suction while ET tube is withdrawn.
- 8 Discard after use.

### GENERAL INFORMATION

#### Product Materials

Acrylic  
This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

Length: 83 mm  
ET tube connection opening: 15 mm (I.D.)  
Thumb port opening: 10 mm (I.D.)

#### Packaging

Individually packaged: Yes  
Package type: Polyethylene bag  
Dispenser box: Yes

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Single patient use only.



For more information or to order your **FREE** sample call  
**1-800-966-0500** or visit **[neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2021 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



Made in USA

M560 Rev D