


NeoPulse[™]

DISPOSABLE PULSE OXIMETER WRAP

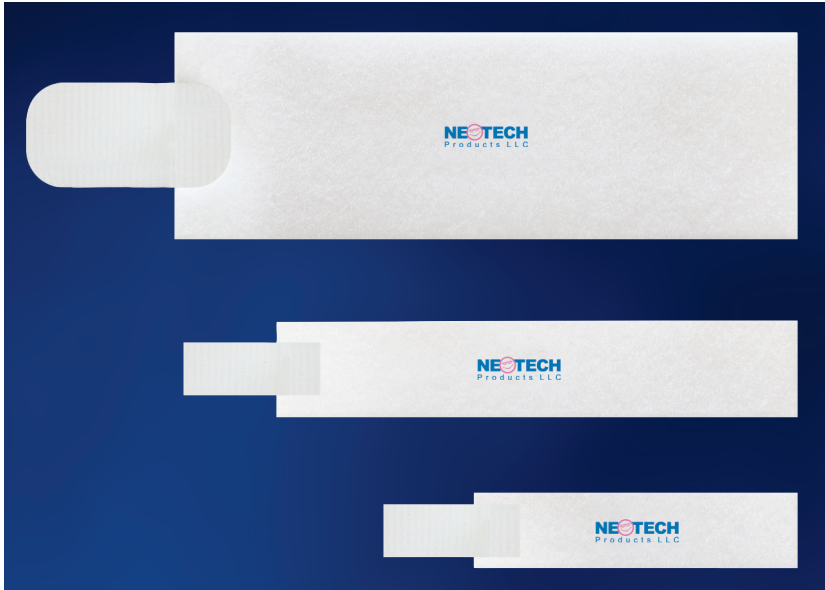

FEATURES & BENEFITS

- Soft, breathable, skin friendly NeoFoam[®] wicks perspiration
- Lightweight, form fitting and adjustable
- Designed for comfort and stability
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

| CAT. # | SIZE | DIMENSIONS | QTY/UNIT |
|--------|---------|------------|----------|
| N781 | Small | 3/4" x 3" | 100/box |
| N782 | Large | 1" x 5" | 100/box |
| N783 | X-Large | 2" x 6" | 100/box |



Made in USA

NeoPulse™

DISPOSABLE PULSE OXIMETER WRAP

INDICATIONS FOR USE

The NeoPulse is intended to hold a pulse oximeter sensor in place. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

- 1 Place pulse oximeter sensor per hospital protocol.
- 2 Wrap NeoPulse around foot and pulse oximeter sensor and gently fasten hook tab to wrap.

Note: May be trimmed for proper fit. Ensure that wrap is not too tight. Hook tab should completely overlap wrap.

- 3 Check foot after wrapping to ensure adequate circulation.
- 4 Change the NeoPulse every 5 days or per hospital protocol, whichever is sooner.
Replace immediately if visibly worn or soiled.

GENERAL INFORMATION

Product Materials

NeoFoam, hook fabric
This product is not made with natural rubber latex or plasticizer DEHP.

Size

N781 Small: 75 mm x 20 mm
N782 Large: 125 mm x 25 mm
N783 XL: 150 mm x 50 mm

Packaging

Individually packaged: Yes
Package type: Polyethylene bag
Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

Boxes are printed with environmentally friendly ink on recyclable material.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.
Single patient use only.



For more information or to order your **FREE** sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

