


NeoSpoon[™]


DISPOSABLE MEASURING SPOON

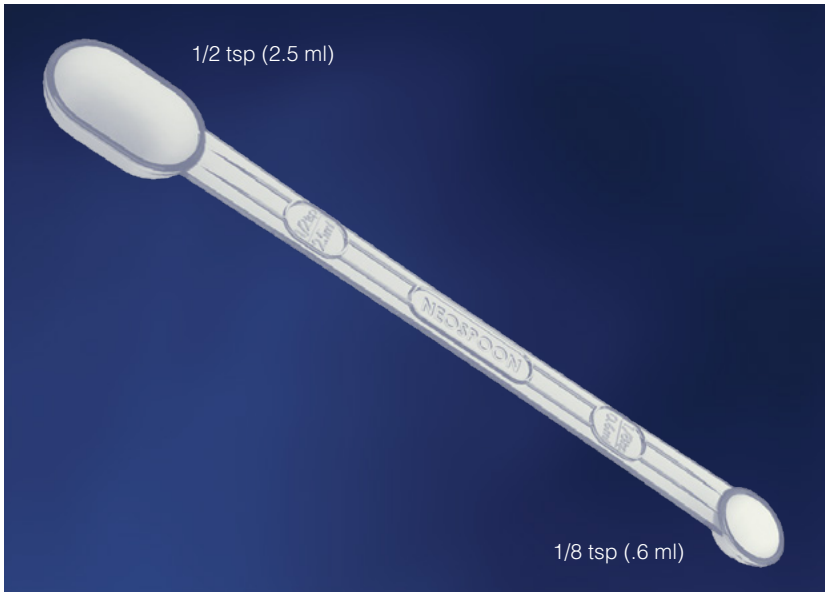

FEATURES & BENEFITS

- Dual spoon design (1/8 tsp and 1/2 tsp)
- Engineered for quick, accurate measuring
- Dispense exact volume in multiples of 1/8 tsp
- Ideal for feeding preparation
- Environmentally friendly, biodegradable material
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT. #	VOLUME	QTY/UNIT
N761	1/8 & 1/2 tsp	100/box

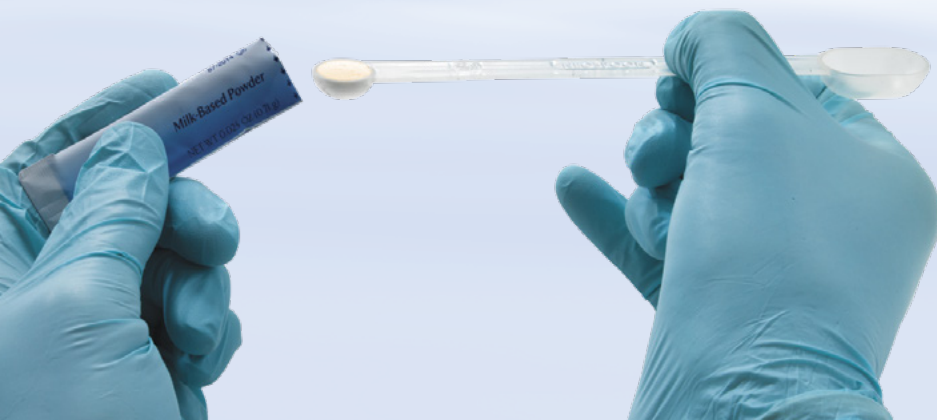

Made in USA

NeoSpoon™

DISPOSABLE MEASURING SPOON

INDICATIONS FOR USE

The NeoSpoon is intended for use as a measuring device.

DIRECTIONS FOR USE

- 1 Scoop and level substance.

Note: Leveling is essential for accurate measurement.

- 2 Measure desired volume of single substance.

Note: Do not use single NeoSpoon for multiple substances.

- 3 Dispose after each use.

Note: If used to measure hazardous substance, dispose in proper hazardous waste container.

GENERAL INFORMATION

Product Materials

NatureWorks Poly lactide Resin

This product is not made with natural rubber latex or plasticizer DEHP.

Size

Length: 168 mm

Volume: 1/8 tsp and 1/2 tsp

Packaging

Individually packaged: Yes

Package type: Polyethylene bag

Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

Boxes are printed with environmentally friendly ink on recyclable material.

CAUTIONS & WARNINGS

If used to measure hazardous substance, dispose in proper hazardous waste container.

Single substance use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355



Made in USA