

## NeoSmile<sup>™</sup>

### TEMPERATURE PROBE COVER



#### FEATURES & BENEFITS

- Skin friendly NeoBond Hydrocolloid base
- Designed for long lasting adhesion
- Reflective foil helps protect against ambient and radiant heat
- Cute smiley face design
- Not made with natural rubber latex or plasticizer DEHP

| CAT. # | SIZE        | QTY/UNIT |
|--------|-------------|----------|
| N731   | 1" Diameter | 150/box  |



# NeoSmile™

## TEMPERATURE PROBE COVER

### INDICATIONS FOR USE

The NeoSmile is intended to cover and secure a temperature probe.

### DIRECTIONS FOR USE

#### Preparation

Clean and dry skin where NeoSmile will be placed per hospital protocol.

*Note: It is important for skin to be as clean and dry as possible. NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.*

DO NOT USE ALCOHOL.

#### Application

- 1 Before removing plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.
- 2 Peel and discard liner. Place NeoSmile over probe and skin and hold in place for 60 seconds to ensure proper adhesion.

#### Removal

- 1 Replace NeoSmile every 5 days or per hospital protocol, whichever is sooner.
- 2 Saturate NeoSmile with water or saline.
- 3 Slowly peel NeoSmile away from skin as you swab the skin with water or saline.

### GENERAL INFORMATION

#### Product Materials

Reflective foil (gold mylar), NeoBond Hydrocolloid  
This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

Diameter: 25 mm

#### Packaging

Standard box, no dispense feature

Perforated liner

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material*

### CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Do not reuse or reapply.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit **[neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2022 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



M590 Rev D