


NeoLead[®] ECG ELECTRODES

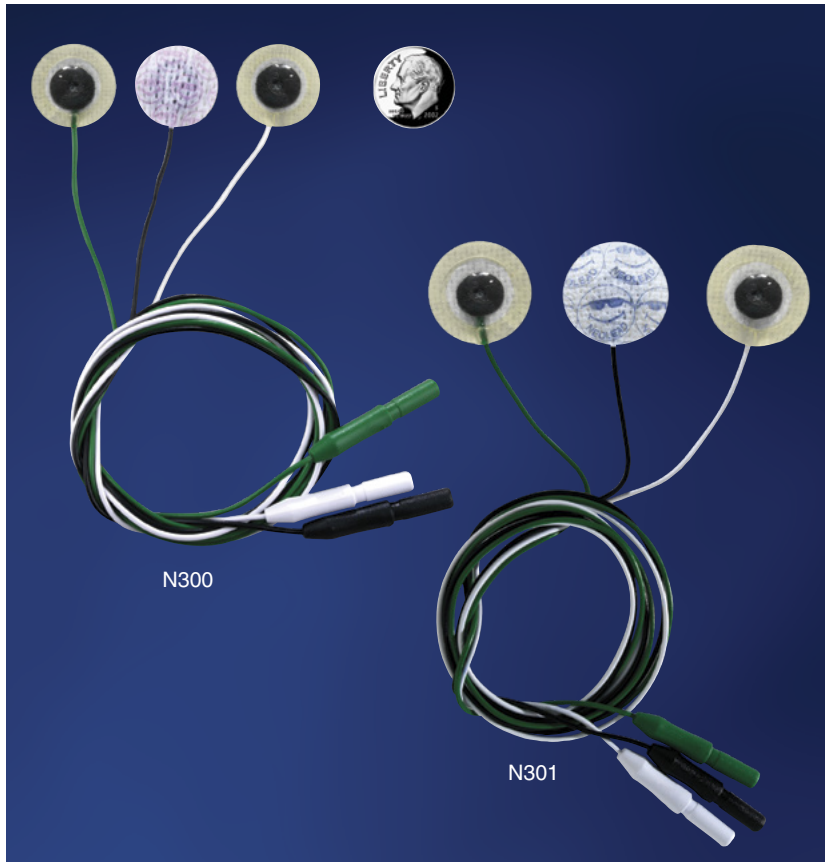

FEATURES & BENEFITS

- Pre-wired electrodes
- Hydrogel center
- NeoBond[®] Hydrocolloid border designed to:
 - prevent hydrogel from oozing or drying prematurely
 - provide sensor stability for better tracings
 - minimize slipping
- Multi-colored wires for easier identification
- Not made with natural rubber latex or plasticizer DEHP
- Sets of three individually packaged

CAT. #	ITEM	SIZE	QTY/UNIT
N300	Mini	3/4" circle	20 sets/box
N301	Standard	1" circle	20 sets/box



U.S. Patent #6,795,722

Clinician Invented

Made in USA

NeoLead®

ECG ELECTRODES

INDICATIONS FOR USE

The NeoLead® ECG Electrodes are intended for electrocardiogram (ECG) monitoring. They are intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Preparation



Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Electrodes will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

- 1 Before removing plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.
- 2 One at a time, peel each lead from plastic liner, place the lead on the skin, and hold in place for 60 seconds to ensure proper adhesion.
Discard the plastic liner.

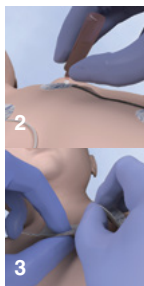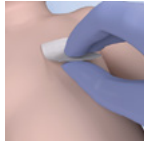

Removal

- 1 Replace NeoLead every 5 days or per hospital protocol, whichever is sooner.
- 2 Saturate the lead with water or saline.
- 3 Slowly peel the lead away from the skin as you swab the skin with water or saline.

GENERAL INFORMATION

Product Materials

Electrode: Silver chloride, hydrogel, NeoBond Hydrocolloid, non-woven fabric
Wire: Copper, polypropylene
This product is not made with natural rubber latex or plasticizer DEHP.

Size

N300 diameter: 19 mm
N301 diameter: 26 mm
Wire length: 61 cm

Packaging

Sets individually packaged
Package type: Pouch
Dispenser box

Visit neotechproducts.com for the most up-to-date information.

Pouches are made from paper, polyethylene and foil. Boxes are printed with environmentally friendly ink on recyclable material.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

NeoLead® ECG Electrodes should only be used while patient is under continuous direct supervision of healthcare professionals.

Electrodes will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Do not use NeoLead (N300, N301) during x-ray or MR procedures.

Discontinue immediately if skin irritation occurs.

Disposable. Do not reuse.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

