


NeoBond[®]

HYDROCOLLOID ADHESIVE

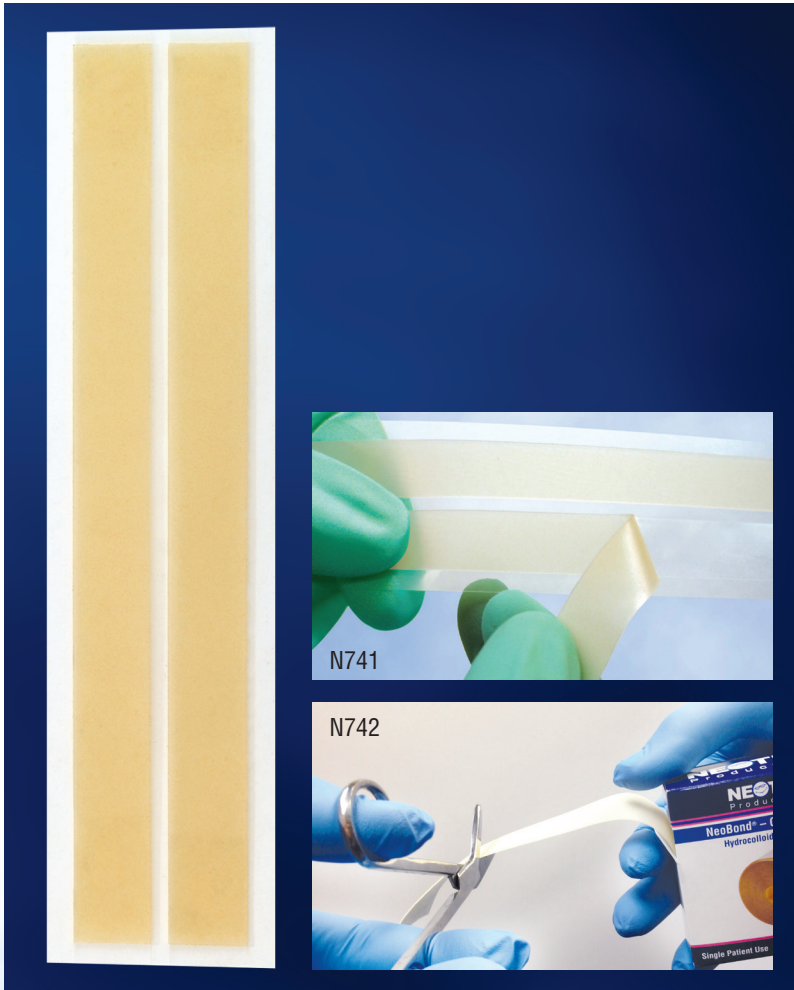

FEATURES & BENEFITS

- Skin friendly alternative to acrylic tape
- Developed for long term attachment
- Ideal for use with limb boards
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	SIZE	QTY/UNIT
N741	1/2" x 6" Strips	100/box*
N742	1" x 15' Roll	1/box

*50 sets of 2



Made in USA

NeoBond®

HYDROCOLLOID ADHESIVE

INDICATIONS FOR USE

NeoBond Hydrocolloid Adhesive is intended for general use as a medical adhesive.

DIRECTIONS FOR USE

Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

- 1 Before removing liner, warm NeoBond in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.
- 2 Before removing liner, warm NeoBond in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Removal

- 1 Replace NeoBond every 5 days or per hospital protocol, whichever is sooner.
- 2 Saturate NeoBond with water or saline.
- 3 Slowly peel NeoBond away from skin as you swab with water or saline.

GENERAL INFORMATION

Product Materials

NeoBond Hydrocolloid, polyester silicone coating, polystyrene backing
This product is not made with natural rubber latex or plasticizer DEHP.

Size

N741 pre-cut strip: 1/2 in. x 6 in.
N742 roll: 1 in. x 15 ft.

Packaging

N741 Strips: Strips of two on plastic liner
N742 Roll: Continuous roll in dispenser box with coated paper liner

Visit neotechproducts.com for the most up-to-date information or to download.

Boxes are printed with environmentally friendly ink on recyclable material.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

