


## NeoBond<sup>®</sup>

### HYDROCOLLOID ADHESIVE

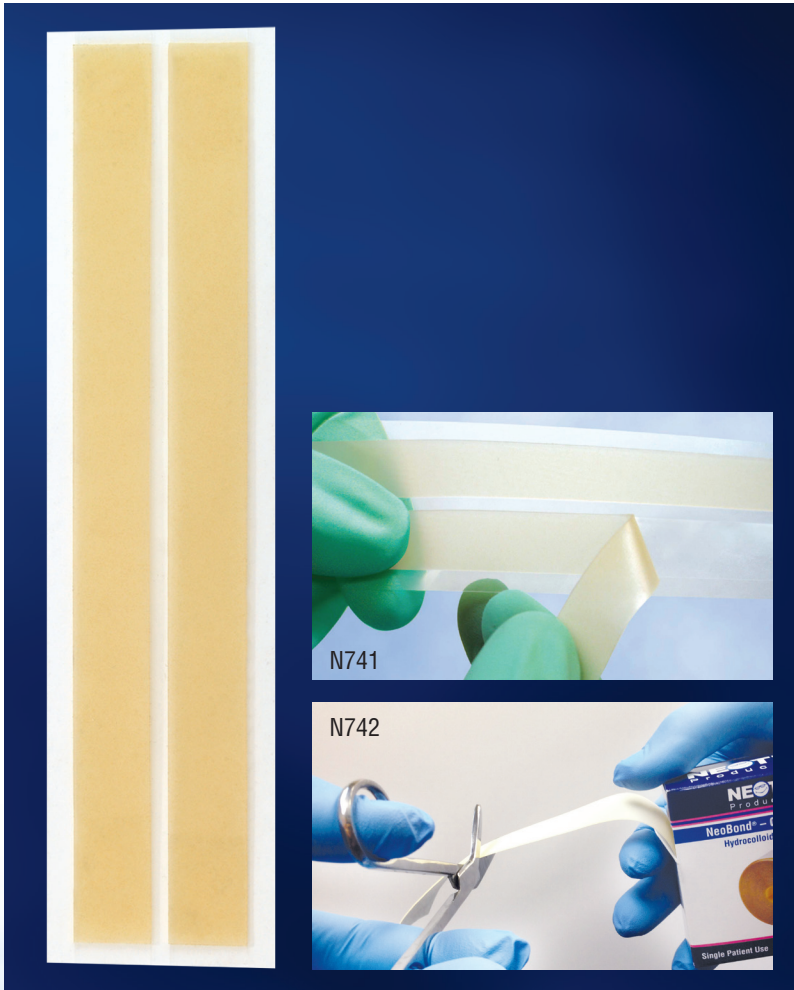

#### FEATURES & BENEFITS

- Skin friendly alternative to acrylic tape
- Developed for long term attachment
- Ideal for use with limb boards
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	SIZE	QTY/UNIT
N741	1/2" x 6" Strips	100/box*
N742	1" x 15' Roll	1/box

\*50 sets of 2



# NeoBond®

## HYDROCOLLOID ADHESIVE

### INDICATIONS FOR USE

NeoBond Hydrocolloid Adhesive is intended for general use as a medical adhesive.

### DIRECTIONS FOR USE

#### Preparation

Clean and dry skin where NeoBond will be placed per hospital protocol.

*Note: It is important for skin to be as clean and dry as possible. NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.*

DO NOT USE ALCOHOL.

#### Application

- 1 Before removing from plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.
- 2 Peel from liner. Wrap or apply NeoBond over skin and product (e.g. feeding tube, limb board) and hold in place for 60 seconds to ensure proper adhesion. Discard liner when done.

#### Removal

- 1 Replace NeoBond every 5 days or per hospital protocol, whichever is sooner.
- 2 Saturate NeoBond with water or saline.
- 3 Slowly peel NeoBond away from skin as you swab the skin with water or saline.

### GENERAL INFORMATION

#### Product Materials

NeoBond Hydrocolloid, polyester silicone coating, polystyrene backing

This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

N741 pre-cut strip: 1/2 in. x 6 in.

N742 roll: 1 in. x 15 ft.

#### Packaging

N741 Strips: Strips of two on plastic liner

N742 Roll: Continuous roll in dispenser box with coated paper liner

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit **[neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2022 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



M608 Rev F