

# Hold-A-Line™

## CATHETER AND TUBING HOLDER



### FEATURES & BENEFITS

- Skin friendly NeoBond Hydrocolloid base
- Fold over flap designed to relieve stress on tubes and help prevent dislodgement
- Unique shape allows for protruding umbilical stump
- Flap designed to be opened and closed repeatedly
- Clear flap allows for better visibility
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	ITEM	QTY/UNIT
N705	Standard	40/box



Made in USA

# Hold-A-Line™

## CATHETER AND TUBING HOLDER

### INDICATIONS FOR USE

The Hold-A-Line is intended to anchor and support umbilical catheters and tubing. It is intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

#### Preparation

After catheter or tube is in place, clean and dry skin per hospital protocol.

*Note: It is important for skin to be as clean and dry as possible. Hold-A-Line will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.*

DO NOT USE ALCOHOL.

#### Application

- 1 Before removing plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.
- 2 Discard white, paper liner, then peel and discard clear, plastic liner from hydrocolloid base.
- 3 Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.
- 4 Peel and discard frosted liner and adhere catheter or tube to base.
- 5 Fold flap over catheter or tube, align with base and press firmly together, enclosing catheter or tube between adhesives.
- 6 Lift flap as needed to reposition or remove catheter or tube.

#### Removal

- 1 Replace Hold-A-Line every 5 days or per hospital protocol, whichever is sooner.
- 2 Lift flap to remove catheter or tube.
- 3 Saturate Hold-A-Line with water or saline.
- 4 Slowly peel Hold-A-Line away from skin as you swab with water or saline.

### GENERAL INFORMATION

#### Product Materials

NeoBond Hydrocolloid, acrylic adhesive (does not contact skin), mylar film

This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

Closed: 50 mm x 25 mm

#### Packaging

Standard box, no dispense feature

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Hold-A-Line should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Discontinue immediately if skin irritation occurs.

Hold-A-Line will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit **[neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355



Made in USA