


NeoSucker[®] Curved

ORAL AND NASAL SUCTION DEVICE

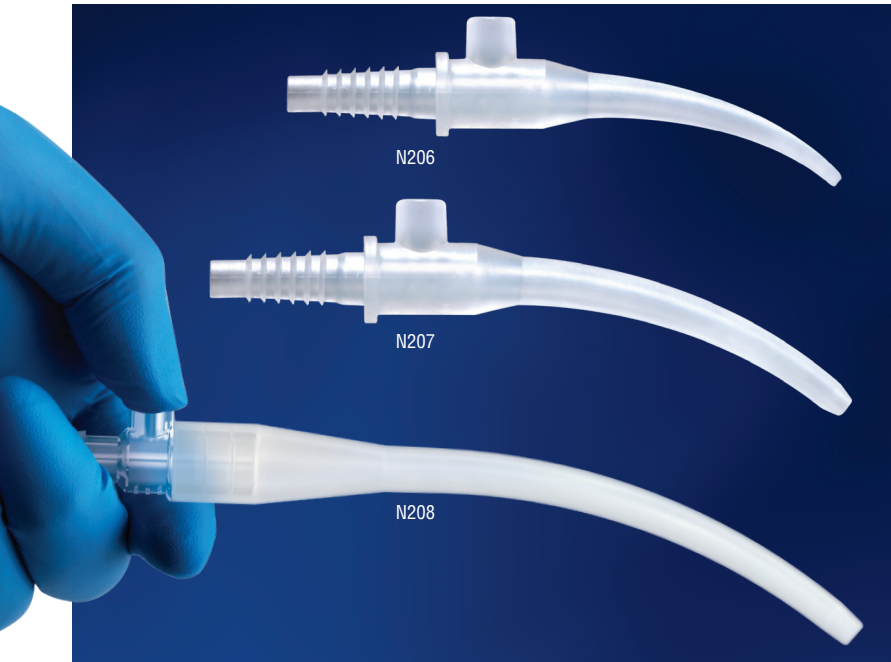

FEATURES & BENEFITS

- Anatomically curved shape
- Designed for single handed suctioning
- Thumb port for intermittent suctioning
- Soft, flexible tip similar to a bulb syringe
- N208 XL ideal for older or larger pediatric patients
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT. #	SIZE	QTY/UNIT
N206	Preemie	50/box
N207	Standard	50/box
N208	XL	50/box



U.S. Patent
#7,635,361; #D590,056;
#6,958,050; #D474,269



NeoSucker® Curved

ORAL AND NASAL SUCTION DEVICE

INDICATIONS FOR USE

The NeoSucker is indicated for use as a tip for suction tubing to aspirate fluids from the mouth and nares.

DIRECTIONS FOR USE

- 1 Connect barbed end of NeoSucker to suction line.
- 2 Set suction to appropriate pressure per hospital protocol or physician recommendation.
- 3 Block thumb port with thumb to begin suctioning.
- 4 Suction mouth or nares with intermittent suctioning. Remove thumb from thumb port to stop suctioning. Block thumb port again to continue suctioning.
- 5 Suction sterile water through NeoSucker after each use to clear debris.
- 6 Replace NeoSucker every 24 hours or per hospital protocol, whichever is sooner.

GENERAL INFORMATION

Product Materials

N206 and N207: TPE (thermoplastic elastomer)

N208: TPE, acrylic

This product is not made with natural rubber latex or plasticizer DEHP.

Size

N206 Preemie

Tip opening diameter: 2 mm

Thumb port opening diameter: 5 mm

N207 Standard

Tip opening diameter: 3 mm

Thumb port opening diameter: 5 mm

N208 XL

Tip opening diameter: 4 mm

Thumb port opening diameter: 6 mm

Packaging

Individually packaged: Yes

Package type: Polyethylene bag

Dispenser box: N206/N207 Yes, N208 No

Visit neotechproducts.com for the most up-to-date information.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

For oral and nasal use only.

Inserting the NeoSucker too far into the mouth or nares can cause tissue trauma.

The NeoSucker should remain free of debris.

Single patient use only.



**For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2022 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



M627 Rev H