


NeoHeart[®]

ECG PAD

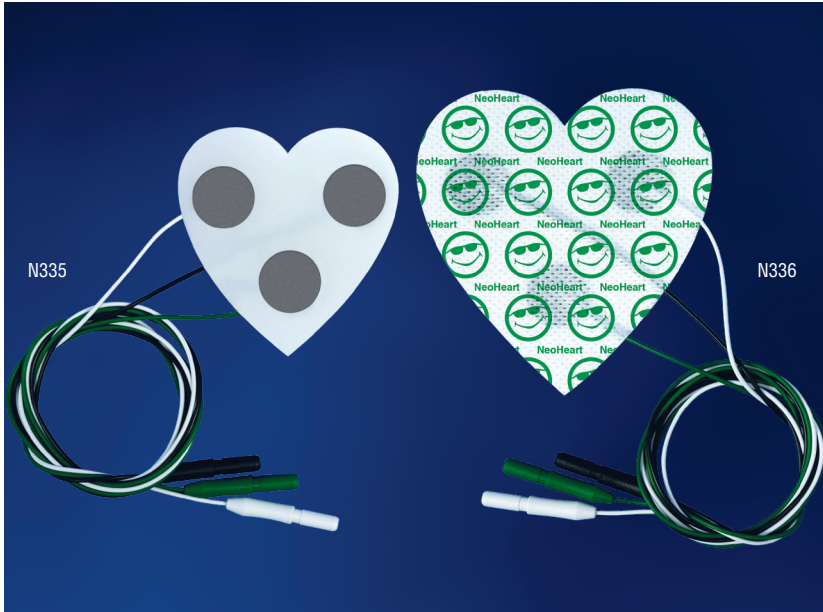

FEATURES & BENEFITS

- 3-in-1 pre-wired electrode array
- Pre-measured sensor position
- Radiolucent material nearly invisible on x-rays
- One touch application ideal for use in delivery room
- Larger surface area for enhanced adherence
- Engineered for back application, allowing chest to remain exposed
- Multi-colored wires for easier identification
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

| CAT. # | SIZE | QTY/UNIT |
|--------|-------|-------------|
| N335 | Small | 20 sets/box |
| N336 | Large | 20 sets/box |

Patent Pending



NeoHeart®

ECG PAD

INDICATIONS FOR USE

NeoHeart is a set of three attached ECG electrodes used on neonates, infants, and pediatric patients.

DIRECTIONS FOR USE

Application

- 1 Wipe and remove any excess moisture from where NeoHeart will be applied.
- 2 Peel and discard liner. Apply immediately after liner is removed.
- 3 Place NeoHeart on patient's back and connect to ECG monitor cable. The pad should be oriented so the point of the heart is pointing toward lower extremities.
- 4 Confirm quality of ECG tracing.

Note: if QRS is stable but appears small or dampened, consider changing to a different lead on the monitor or increasing the size of the waveform. If small or dampened waveform persists, application to the patient's chest may result in a stronger waveform.

CAUTION: If placed on patient's chest, the white wire electrode will be on patient's left side. Insert into monitor accordingly.

Removal

- 1 Remove NeoHeart within 24 hours.
- 2 Saturate the exposed (fabric) side of the NeoHeart with water or saline.
- 1 Slowly peel NeoHeart away from skin as you continuously swab the skin and adhesive side of the NeoHeart with water or saline.

GENERAL INFORMATION

Product Materials

Heart: Non-woven fabric, medical grade acrylic
Electrode: Carbon, silver silver chloride, hydrogel
Wire: Carbon, polypropylene
Made with natural rubber latex: No
Made with plasticizer DEHP: No

Size

N335: 61 x 63.5 mm
N336: 82 x 84 mm
Wire length: 61 cm

Packaging

Individually packaged: Yes
Package type: Pouch
Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

Pouches are made from paper, polyethylene and foil.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

NeoHeart ECG Pad should only be used while patient is under continuous direct supervision of healthcare professionals.

NeoHeart is designed for back application. If placed on patient's chest, the white wire electrode will be on patient's left side. Insert into monitor accordingly.

Do not apply to open wounds or broken skin.

Discontinue immediately if skin irritation occurs.

Discard NeoHeart 24 hours after opening pouch.

Disposable. Do not reuse.

Single patient use only.



For more information or to order your **FREE** sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2023 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



M638 Rev E