



**NeoHug<sup>®</sup>**  
 Utility Device Holder

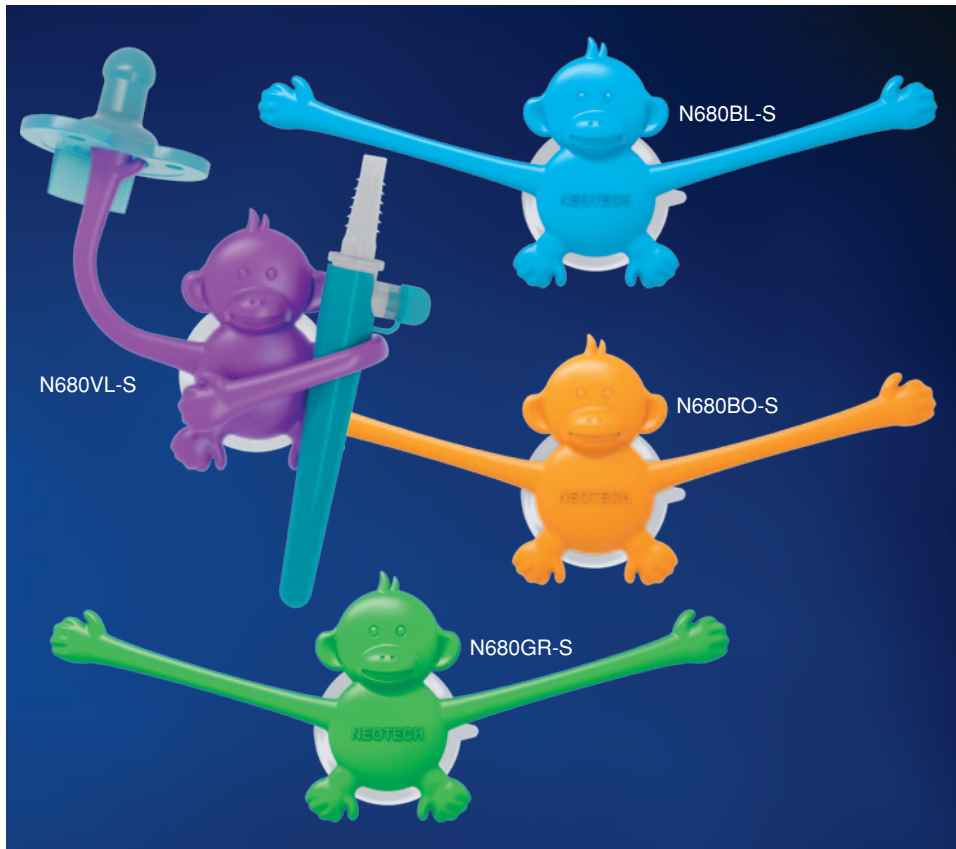

**FEATURES AND BENEFITS**

- Playful chimp design available in five bright colors
- Bendable arms to hold a variety of light weight items
- Ideal for holding pacifier, suction tip, tape and more
- Two attachment options
- C-clip:
  - o Attaches to most IV poles
  - o Snaps on and off
  - o Ribbed design to minimize slipping
  - o Reposition repeatedly with no adhesive residue
- Suction Cup:
  - o Attaches to most smooth surfaces
  - o Large suction cup for strong hold and easy removal
  - o Holds under heat and humidity
  - o Reposition repeatedly with no adhesive residue
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

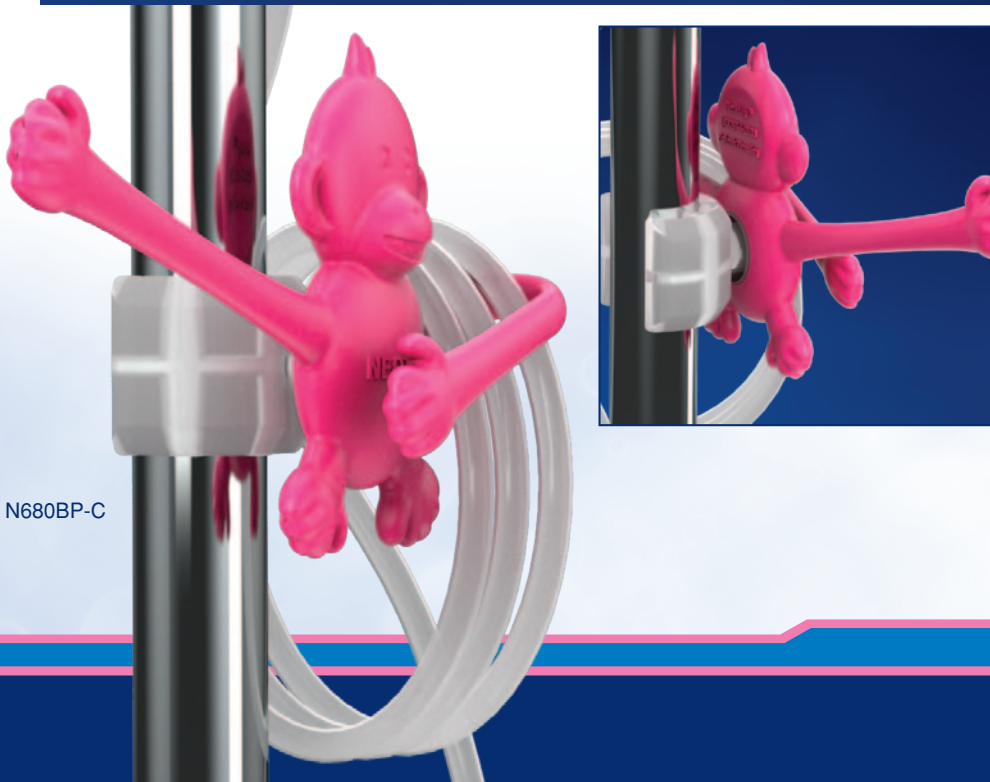

Cat No.	Color	Qty/Unit
N680BL	Blue	25/box
N680BO	Bright Orange	25/box
N680BP	Bright Pink	25/box
N680GR	Green	25/box
N680VL	Violet	25/box
N680A	Assorted (5 of each color)	25/box

C-clip=C, Suction Cup=S  
 (e.g. N680BL-C, N680GR-S)



**Clinician Invented**

Patent Pending

# NeoHug®

## Utility Device Holder

### INDICATIONS FOR USE

---

NeoHug is intended to support, organize or hold non-operating medical devices.

### DIRECTIONS FOR USE

---

#### Step 1

Remove NeoHug from package and discard biodegradable bag.

#### Step 2a (C-clip)

Place NeoHug onto IV pole by pressing center of C-clip (belly) until it snaps into place.

#### Step 2b (Suction)

Affix NeoHug by pressing center of suction cup (belly) firmly against smooth, flat surface.

#### Step 3

Bend and conform arms as needed to hold desired items.

### Removal

#### Step 1

Replace NeoHug every 14 days or per hospital protocol, whichever is sooner.

#### Step 2a (C-clip)

Firmly hold C-clip and pull to slide off. Do not pull on chimp.

#### Step 2b (Suction)

Lift suction cup tab to pop off of surface.

Caution: Replace immediately if visibly soiled or damaged OR if C-clip or suction cup no longer holds effectively.

### GENERAL INFORMATION

---

#### Product Materials

NeoHug (chimp): TPE, ABS, stainless steel

C-clip: ABS

Suction cup: Vinyl

Not made with natural rubber latex.

Not made with plasticizer DEHP.

#### Size

Bendable arm span: 19.5 cm

C-clip: extends 28.5 mm beyond chimp x 25.5 mm tall

Suction cup diameter: 45 mm

#### Weight Limit

Holds up to 113 gr (4 oz) with both arms.

#### Packaging

Individually packaged: Yes

Package type: Biodegradable polyethylene bag

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

### CAUTIONS

---

The suction cup should be kept clean and free of debris and fluids.

Replace NeoHug immediately if visibly soiled or damaged.

Replace if C-clip or suction cup no longer holds effectively.

Single patient use only.

**For more information or to order your FREE sample call  
1-800-966-0500 or visit [neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Parkway • Valencia, CA 91355