

NeoHug[®]

UTILITY DEVICE HOLDER



FEATURES & BENEFITS

- Playful chimp or smiley face design in five bright colors
- Bendable arms to hold a variety of light weight items
- Ideal for holding pacifier, suction tip, tape and more
- Two attachment options
- C-clip:
 - Attaches to 3/4" (19mm) portion of IV poles
 - Snaps on and off
 - Ribbed design to minimize slipping
 - Reposition repeatedly with no adhesive residue
- Suction Cup:
 - Attaches to most smooth surfaces
 - Large suction cup for strong hold and easy removal
 - Holds under heat and humidity
 - Reposition repeatedly with no adhesive residue
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT. #	ITEM	QTY/UNIT
N680A-C	Assorted Chimp C-Clip	25/box
N680A-S	Assorted Chimp Suction	25/box
N681A-C	Assorted Smile C-Clip	25/box
N681A-S	Assorted Smile Suction	25/box

Assorted = 5 of each color



NeoHug®



UTILITY DEVICE HOLDER

INDICATIONS FOR USE

NeoHug is intended to support, organize or hold non-operating medical devices.

DIRECTIONS FOR USE

- 1 Remove NeoHug from package and discard bag.
- 2a **(C-clip)** Place NeoHug onto IV pole by pressing center of C-clip until it snaps into place.
- 2b **(Suction)** Affix NeoHug by pressing center of suction cup firmly against smooth, flat surface.
Note: If mounting in high humidity (i.e. inside incubators), wipe the surface of excess moisture before mounting.
- 3 Bend and conform arms as needed to hold desired items.
- 4 Clean as needed per hospital protocol. Use a new NeoHug for each patient.

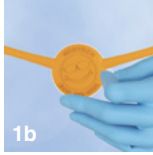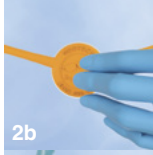

Removal

- 1a **(C-clip)** Firmly hold C-clip and pull to slide off. Do not pull on NeoHug.
- 1b **(Suction)** Lift suction cup tab to pop off of surface.

Caution: Replace immediately if damaged OR if C-clip or suction cup no longer holds effectively.

GENERAL INFORMATION

Product Materials

NeoHug: TPE, ABS, stainless steel
C-clip: ABS
Suction cup: Vinyl
This product is not made with natural rubber latex or plasticizer DEHP.

Size

Bendable arm span: 19.5 cm
C-clip: extends 28.5 mm beyond NeoHug x 25.5 mm tall
Suction cup diameter: 45 mm

Packaging

Individually packaged
Package type: Polyethylene bag

Visit neotechproducts.com for the most up-to-date information.

CAUTIONS & WARNINGS

The suction cup should be kept clean and free of debris and fluids.

Replace NeoHug immediately if damaged.

Replace if C-clip or suction cup no longer holds effectively.

Do not force C-clip onto pole larger than 3/4" diameter.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

