


NeoShades[®]

PHOTOTHERAPY EYE SHIELDS

N720PL

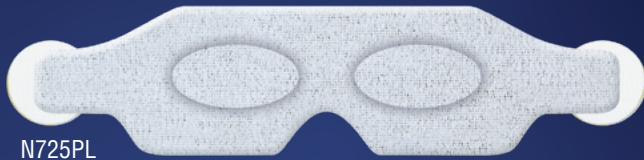

N725PL



N729PL



FEATURES & BENEFITS

- Available with skin friendly NeoBond[®] Hydrocolloid tabs or soft, patented NeoFoam[®] Headstrap to hold NeoShades in place
- Soft, padded eye shields
- Some SKUs feature ocular pockets designed for patient comfort and safety
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT NO.	ITEM	QTY/UNIT
N720PL	Micro	Tabs 50/box
N720PL-H	Micro	Headstrap 50/box
N721PL	Small	Tabs 50/box
N721PL-H	Small	Headstrap 50/box
N722PL	Large	Tabs 50/box
N722PL-H	Large	Headstrap 50/box
With Ocular Pockets		
N724PL	Micro	Tabs 50/box
N725PL	Small	Tabs 50/box
N726PL	Large	Tabs 50/box
N727PL	Micro	Headstrap 50/box
N728PL	Small	Headstrap 50/box
N729PL	Large	Headstrap 50/box

U.S. Patent #8,070,288;
Additional Patents Pending



NeoShades®

PHOTOTHERAPY EYE SHIELDS

INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

With Tabs

Preparation

- 1 Select proper size.

Approximate sizing:
Micro – 28 weeks and under
Small – 29-36 weeks
Large – 37 weeks and up

- 2 Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

- 1 Carefully remove hydrocolloid tabs from eye shields.

Before removing plastic liners, hydrocolloid adhesive tabs must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds

- 2 Peel and discard plastic liners. Place hydrocolloid adhesive tabs on temples and hold in place for 60 seconds to ensure proper adhesion.

- 3 Cover eyes with NeoShades and gently attach to hydrocolloid adhesive tabs.

Note: Leave a portion of the tab exposed for easier removal.

Removal

- 1 Replace NeoShades hydrocolloid adhesive tabs on skin every 5 days or per hospital protocol, whichever is sooner.
- 2 Hold edge of hydrocolloid adhesive tab in place and slowly separate NeoShades from tab.
- 3 Saturate hydrocolloid adhesive tab with water or saline.

- 4 Slowly peel hydrocolloid adhesive tab away from skin as you swab with water or saline.

Repeat for opposite side.

With Headstrap

- 1 Select proper size.

Approximate sizing:
Micro – 28 weeks and under
Small – 29-36 weeks
Large – 37 weeks and up

- 2 Place headstrap behind head with smooth side of foam against head.

- 3 Cover eyes with NeoShades.

- 4 Gently attach hook fabric on end of headstrap to NeoShades. Repeat for opposite side.

Note: Straps can be trimmed for proper fit.



— CONTRAINDICATIONS, CAUTIONS & WARNINGS —

The NeoShades with Tabs are contraindicated for patients with facial deformation, skin infection or excoriated skin areas that would prevent the proper placement of the tabs.

The NeoShades are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

When using NeoShades intermittently, always assess the integrity of the hydrocolloid tabs whenever the eye shields are removed. If the hydrocolloid tabs are pulling away from the skin, remove per the directions for use and apply new hydrocolloid tabs before resuming phototherapy.

Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Discontinue use immediately if skin irritation occurs.

Single patient use only.



For more information or to order your FREE sample call

1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355



EMERGO EUROPE
Westervoortsedijk 60
6827 AT, Arnhem
The Netherlands



PPE Regulation (EU) 2016/425
Notified Body: BSI Group (2797),
Say Building, John M. Keynesplein 9
1066 EP Amsterdam, Netherlands