NEOTECH[®]
Products LLC

EZ-Hold[®] Tubing and Cannula Holder

REF Catalog No.

N756
N757
N758

DIRECTIONS FOR USE
INSTRUCTIONS D'UTILISATION
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
GEBRAUCHSANWEISUNG
NÁVOD K POUŽITÍ
ZALECENIA DOTYCZĄCE
UŻYTKOWANIA
إرشادات الاستخدام

 Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EC REP EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

CE

Made in USA

D483 Rev F

English
EZ-Hold[®]
Tubing and Cannula Holder

INDICATIONS FOR USE

The EZ-Hold is intended to anchor and support breathing tubes.

DIRECTIONS FOR USE

Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing liners, warm EZ-Hold in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Step 2 Discard white, paper liner, then peel and discard clear, plastic liner from hydrocolloid base

Step 3 Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.

Step 4 Peel and discard second liner and fold flap over tubing.

Note: Lift flap as needed to reposition or remove tubing.

Removal

Step 1 Replace EZ-Hold every 5 days or per hospital protocol, whichever is sooner.

Step 2 Lift flap to remove tubing.

Step 3 Saturate EZ-Hold with water or saline.

Step 4 Slowly peel EZ-Hold away from skin as you swab with water or saline.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

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