NEOTECH[®]
Products LLC

NeoHug[®] Utility Device Holder

[REF] Catalog No.	DIRECTIONS FOR USE INSTRUCTIONS D'UTILISATION ISTRUZIONI PER L'USO INSTRUCCIONES DE USO INSTRUÇÕES DE UTILIZAÇÃO GEBRAUCHSANWEISUNG NÁVOD K POUŽITÍ ZALECENIA DOTYCZĄCE UZYTEKOWANIA ANVÄNDARINSTRUKTIONER KÄYTTÖOHJEET إرشادات الاستخدام
N680-C N680-S N681-C N681-S	
	

[EC REP]	EMERGO EUROPE Prinsessegracht 20 2514 AP, The Hague, The Netherlands
-----------------	---

[MD]	Medical Device
-------------	----------------

CE

English
NeoHug[®]
Utility Device Holder

INDICATIONS FOR USE

NeoHug is intended to support, organize or hold non-operating medical devices.

DIRECTIONS FOR USE

Step 1 Remove NeoHug from package and discard bag.

Step 2a (C-clip)

Place NeoHug onto IV pole by pressing center of C-clip until it snaps into place.

Step 2b (Suction)

Affix NeoHug by pressing center of suction cup firmly against smooth, flat surface.

Note: If mounting in high humidity (i.e. inside incubators), wipe the surface of excess moisture before mounting.

Step 3

Bend and conform arms as needed to hold desired items.

Removal

Step 1

Replace NeoHug every 14 days or per hospital protocol, whichever is sooner.

Step 2a (C-clip)

Firmly hold C-clip and pull to slide off. Do not pull on NeoHug.

Step 2b (Suction)

Lift suction cup tab to pop off of surface.

[CAUTIONS]
The suction cup should be kept clean and free of debris and fluids.
Replace immediately if visibly soiled or damaged.
Replace if C-clip, suction cup, or headrail adapter no longer holds effectively.
Do not force C-clip onto pole larger than 3/4” diameter.
Single patient use only.

[EUROPEAN UNION]
European Union: MDR EU 2017/745
<i>Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.</i>

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