NeoShades[®] with Tabs Phototherapy Eye Shields

REF Catalog No.	DIRECTIONS FOR USE
N720 Micro	INSTRUCTIONS D'UTILISATION
N721 Small	ISTRUZIONI PER L'USO
N722 Large	INSTRUCCIONES DE USO
N724 Micro	INSTRUÇÕES DE UTILIZAÇÃO
N725 Small	GEBRAUCHSANWEISUNG
N726 Large	NÁVOD K POUŽITÍ
	ZALECENIA DOTYCZĄCE
	UŻYTKOWANIA
	إرشادات الاستخدام

Manufactured by
NEOTECH PRODUCTS LLC
26430 Witherspoon Pkwy, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EC REP EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

D457 Rev G

PPE Regulation (EU) 2016/425

Notified Body: BSI Group (2797),
Say Building, John M. Keynesplein 9
1066 EP Amsterdam, Netherlands.

English

NeoShades[®] with Tabs
Phototherapy Eye Shields

INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Preparation

Step 1 Select proper size.

Step 2 Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing liners, carefully remove tabs from NeoShades and warm tabs in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Step 2 Peel and discard liners. Place tabs on skin and hold in place for 60 seconds to ensure proper adhesion.

Step 3 Cover eyes with NeoShades and gently attach to tabs.

Note: Leave a portion of the tab exposed for easier removal.

Removal

Step 1 Hold edge of tab in place and slowly separate NeoShades from tab.

Step 2 Saturate tab with water or saline.

Step 3 Slowly peel tab away from skin as you swab with water or saline.

Repeat for opposite side.

CONTRAINDICATIONS, CAUTIONS & WARNINGS

The NeoShades with Tabs are contraindicated for patients with facial deformation, skin infection or excoriated skin areas that would prevent the proper placement of the tabs. They are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Tabs should always be removed when NeoShades are taken off the patient. Do not reuse.

Discontinue immediately if skin irritation occurs.

Single patient use only.

General transportation and storage practices are applicable.

Declaration of conformity can be viewed on the Neotech website: www.neotechproducts.com

Product expiration date is printed on the immediate packaging as well as the dispenser box of the product.

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