


NeoShades[®] with Tabs

Phototherapy Eye Shields

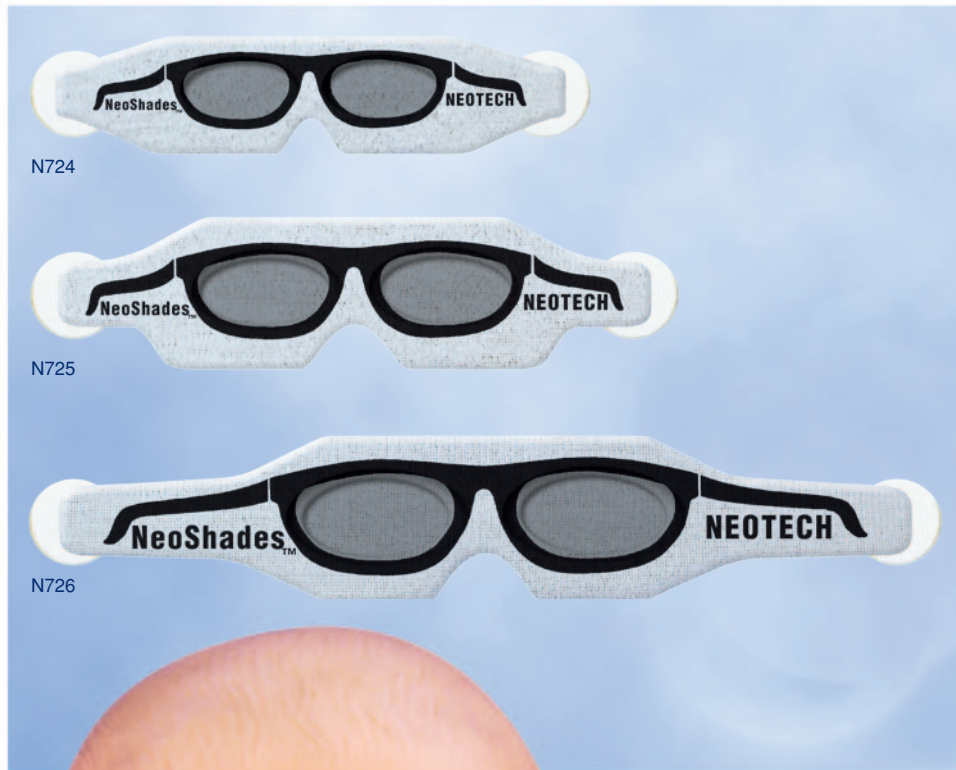

FEATURES AND BENEFITS

- Soft, padded eye shield material with cute sunglasses design
- Ocular pockets designed for patient comfort and safety (N724, N725, N726)
- Skin friendly NeoBond[®] Hydrocolloid tabs hold NeoShades in place
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Item	Qty/Unit
N720	Micro	50/box
N721	Small	50/box
N722	Large	50/box
With Ocular Pockets		
N724	Micro	50/box
N725	Small	50/box
N726	Large	50/box



U.S. Patent #8,070,288;
 Additional Patents Pending

NeoShades® with Tabs

Phototherapy Eye Shields

INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Preparation

Step 1

Select proper size.

Step 2

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1

Before removing liners, carefully remove tabs from NeoShades and warm tabs in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Step 2

Peel and discard liners. Place tabs on skin and hold in place for 60 seconds to ensure proper adhesion.

Step 3

Cover eyes with NeoShades and gently attach to tabs.

Note: Leave a portion of the tab exposed for easier removal.

Removal

Step 1

Hold edge of tab in place and slowly separate NeoShades from tab.

Step 2

Saturate tab with water or saline.

Step 3

Slowly peel tab away from skin as you swab with water or saline.

Repeat for opposite side.

GENERAL INFORMATION

Light Blocking

99.9%

Product Materials

NeoShades: Nylon, polyurethane
Tabs: NeoBond Hydrocolloid, hook fabric
Made with natural rubber latex: No
Made with plasticizer DEHP: No

Size

N720, N724 Micro: 108 mm
(fits head circumference < 26 cm)
N721, N725 Small: 127 mm
(fits head circumference 26-34 cm)
N722, N726 Large: 178 mm
(fits head circumference > 34 cm)

Packaging

Individually packaged: Yes
Package type: Polyethylene bag
Dispenser box: Yes (N720, N721, N722 only)

Visit neotechproducts.com for the most up-to-date information.

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CONTRAINDICATIONS, CAUTIONS & WARNINGS

The NeoShades with Tabs are contraindicated for patients with facial deformation, skin infection or excoriated skin areas that would prevent the proper placement of the tabs. They are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Tabs should always be removed when NeoShades are taken off the patient. Do not reuse.

Discontinue immediately if skin irritation occurs.

Single patient use only.

For more information or to order your FREE sample call

1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA



EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague
The Netherlands



PPE Regulation (EU) 2016/425
Notified Body: BSI Group (2797),
Say Building, John M. Keynesplein 9
1066 EP Amsterdam, Netherlands