

About NeoBond[®] Hydrocolloid

Important, Please Read!

Skin friendly NeoBond Hydrocolloid is a protective adhesive that helps decrease the risk of damage to the fragile epidermis of the neonate. *AWHONN Neonatal Skin Care (Third Edition, 2013)* recommends that hydrocolloids be used in place of other adhesives whenever possible.

Heat is the catalyst for hydrocolloid. The following steps are necessary to ensure the effectiveness of hydrocolloid based products.



PREPARATION

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Products will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.



APPLICATION

Before removing liners, warm hydrocolloid in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Remove liners, apply hydrocolloid base to skin and hold in place for 60 seconds to ensure proper adhesion.



REMOVAL

Replace product every 5 days or per hospital protocol, whichever is sooner.

Saturate hydrocolloid with water or saline.

Slowly peel away from skin as you swab with water or saline.

Always refer to the Directions For Use for each individual product.



Made in USA

For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC. • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA