## Neotech ChinStrap™ Respiratory Support Accessory

**REF** Catalog No.

N784 Nano  
N785 Micro  
N786 Small  
N787 Large  
N788 X-Large  
N789 Macro  
N790 Jumbo

**DIRECTIONS FOR USE**  
**INSTRUCTIONS D'UTILISATION**  
**ISTRUZIONI PER L'USO**  
**INSTRUCCIONES DE USO**  
**INSTRUÇÕES DE UTILIZAÇÃO**  
**GEBRAUCHSANWEISUNG**  
**NÁVOD K POUŽITÍ**  
**ZALECENIA DOTYCZĄCE**  
**UŻYTKOWANIA**  
**إرشادات الاستخدام**

 Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

**EC REP** EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands

Made in USA

D459 Rev G

*English*

**Neotech ChinStrap™  
Respiratory Support Accessory**

### INDICATIONS FOR USE

The Neotech ChinStrap is intended to keep a patient's mouth closed during respiratory therapy. It is intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

**Step 1** Select proper size by measuring circumference from center of chin, over the ear, to posterior of head.

**Step 2** Place Neotech ChinStrap across chin with indentation facing neck and smooth, white side against skin.

**Step 3** Position wider, lower portion of split chin strap under chin and upper portion across front of chin.

**Step 4** Pull strap toward posterior of head with ears positioned inside ear openings and gently fasten hook tab to strap. Side marked "LEFT" should be under patient's left ear.

Note: May be trimmed for proper fit. Ensure that strap is not too tight or pressing on ears. Hook tab should completely overlap strap.

**Step 5** Replace every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if worn or visibly soiled.

### CAUTIONS & WARNINGS

The Neotech ChinStrap is not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

Discontinue immediately if skin irritation occurs.

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