


# Neotech Ocular Exam Kit

## Disposable Speculum and Depressor

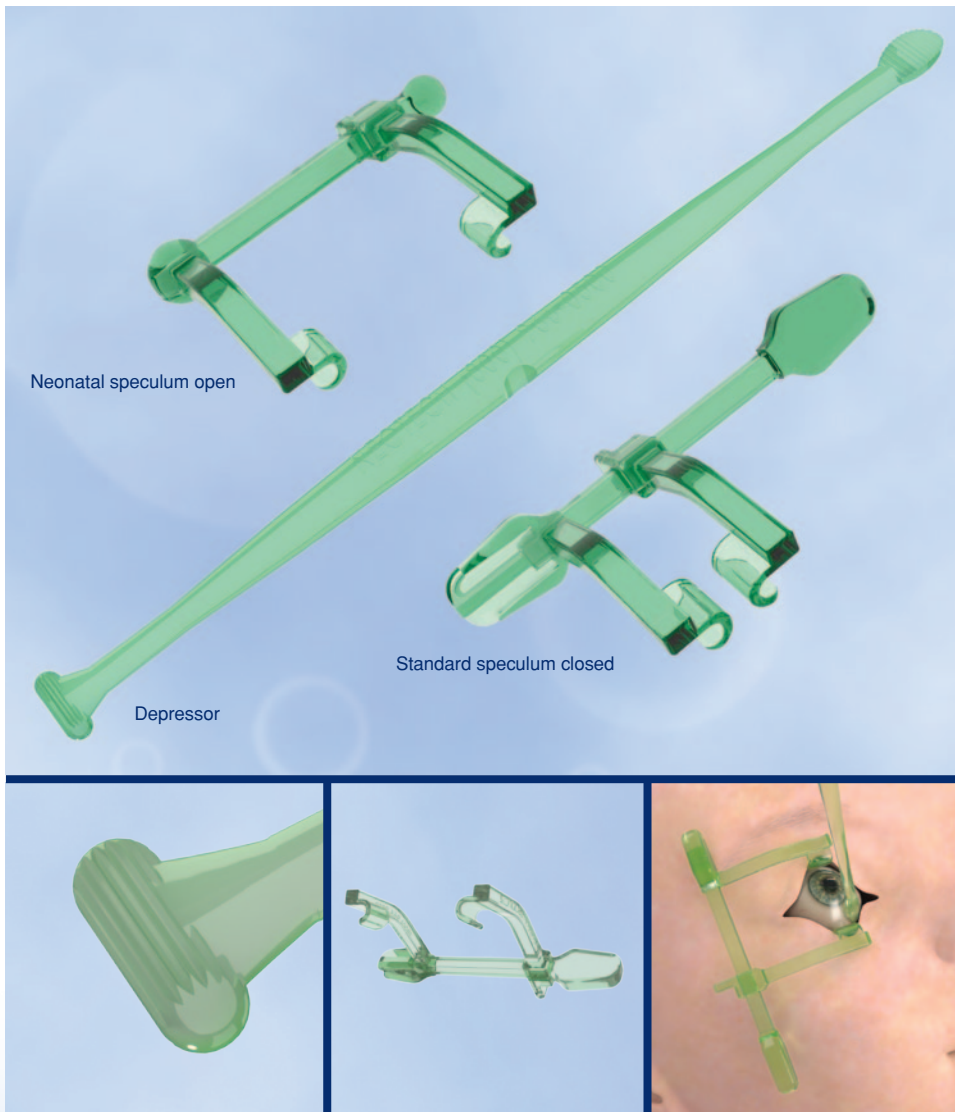

### FEATURES AND BENEFITS

- Disposable, sterile and ready to use
- Unique speculum designed to:
  - Lock in place with eyelid pressure
  - Allow unrestricted view
  - Open and close with ease
- Speculum available in two sizes
- Neonatal speculum (N6501) ideal for ROP exam
- Two-sided depressor with innovative texture intended to gently grip the conjunctiva
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Item	Qty/Unit
N6501	Neonatal Speculum/Depressor	10/box
N6511	Standard Speculum/Depressor	10/box



U.S. Patent #9,289,199

# Neotech Ocular Exam Kit

## Disposable Speculum and Depressor

### INDICATIONS FOR USE

The Neotech Ocular Exam Kit is intended to assist in ocular examinations.

### DIRECTIONS FOR USE

#### Step 1

Move sliding arm to closed position.

#### Step 2

Insert curved blades under margins of upper and lower eyelids.

#### Step 3

Move sliding arm to open eyelids to desired spacing and release.

#### Step 4

Use depressor to assist in visualization.

#### Step 5

Slide sliding arm to closed position and remove curved blades from eyelid margins.

#### Step 6

Discard speculum and depressor.

#### Bilateral Examination

Repeat steps above with second speculum and depressor.

Caution: To prevent cross contamination, use one set of instruments per eye.

### GENERAL INFORMATION

#### Product Materials

Speculum: Thermoplastic resin

Depressor: Thermoplastic resin

Wrap: Paper, polyethylene film

Made with natural rubber latex: No

Made with plasticizer DEHP: No

#### Size

Speculum maximum aperture: 23 mm

Depressor cylindrical head width: 9 mm

Depressor spoon head width: 5.5 mm

#### Packaging

Individually packaged: Yes

Package type: Pouch

Dispenser box: Yes

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Pouch is NXL film and reinforced coated paper.*

*Box is printed with environmentally friendly soy ink on recyclable material.*

### CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

The Neotech Ocular Exam Kit should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

To prevent cross contamination, use one set of instruments per eye.

Excessive pressure to the eye may cause trauma to the conjunctiva.

Do not use if package is open or damaged.

Do not use a single speculum and/or depressor on both eyes.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit [neotechproducts.com](http://neotechproducts.com)

Neotech Products LLC • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA