Ocular Exam Kit

Disposable Speculum and Depressor

[REF] Catalog No.

N6501
N6511

DIRECTIONS FOR USE
INSTRUCTIONS D'UTILISATION
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
GEBRAUCHSANWEISUNG
NÁVOD K POUŽITÍ
ZALECENIA DOTYCZĄCE
UŻYTKOWANIA
إرشادات الاستخدام

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

Made in USA

D482 Rev A

English
Ocular Exam Kit
Disposable Speculum and Depressor

INDICATIONS FOR USE

The Neotech Ocular Exam Kit is intended to assist in ocular examinations.

DIRECTIONS FOR USE

Step 1 Move sliding arm to closed position.

Step 2 Insert curved blades under margins of upper and lower eyelids.

Step 3 Move sliding arm to open eyelids to desired spacing and release.

Step 4 Use depressor to assist in visualization.

Step 5 Slide sliding arm to closed position and remove curved blades from eyelid margins.

Step 6 Discard speculum and depressor.

Bilateral Examination

Repeat steps above with second speculum and depressor.

Caution: To prevent cross contamination, use one set of instruments per eye.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

The Neotech Ocular Exam Kit should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

To prevent cross contamination, use one set of instruments per eye.

Excessive pressure to the eye may cause trauma to the conjunctiva.

Do not use if package is open or damaged.

Do not use a single speculum and/or depressor on both eyes.

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