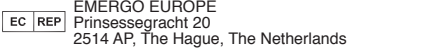
NEOTECH [®] Products LLC	
EZCare™ Softouch Tracheostomy Tube Holder	
REF Catalog No.	DIRECTIONS FOR USE
N9105 5" Single piece	ISTRUZIONI PER L'USO
N9106 6" Single piece	INSTRUCCIONES DE USO
N9107 7" Single piece	INSTRUÇÕES DE UTILIZAÇÃO
N9108 8" Single piece	GEBRAUCHSANWEISUNG
N9109 9" Single piece	NÁVOD K POUŽITÍ
N9110 10" Single piece	ZALECENIA DOTYCZĄCE
N9111 11" Single piece	UZYTEKOWANIA
N925 11" 2-piece	إرشادات الاستخدام
N926 18" 2-piece	

--

--

--

--

English
EZCare™ Softouch Tracheostomy Tube Holder

INDICATIONS FOR USE

The EZCare Softouch is intended to secure a patient’s tracheostomy tube. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Measuring

Select proper size by using EZCare Softouch measuring tape strip provided.

Place start line at flange on left side and wrap behind neck to right flange. If measurement falls between sizes, try larger size first.

Single Piece (N9105-N9111)

Step 1 With smooth, white side facing neck, insert hook tab through flange on one side of plate.

Step 2 Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securement.

Step 3 Wrap EZCare around neck and insert hook tab through flange on other side of plate.

Step 4 Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securement.

Note: Ensure that EZCare is taught, but not overly stretched. Adjust as needed.

Step 5 Replace EZCare Softouch every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if hook tab is worn or visibly soiled.

2-piece (N925-N926)

Step 1 Separate two pieces.

Step 2 With smooth, white side facing neck, insert hook tabs through flanges on each side of plate.

Step 3 Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securement.

Step 4 Wrap both sides of EZCare around neck and bring together.

Step 5 Determine proper length, pull longer piece away from neck and cut excess.

Note: Ensure that skin will not be exposed to hook fabric secure tab.

Step 6 Bring both sides back together and firmly press white hook fabric secure tab to neckband fabric to ensure proper securement.

Note: Ensure that EZCare is taught, but not overly stretched. Adjust as needed.

Step 7 Replace EZCare Softouch every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if hook tab is worn or visibly soiled.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Replace immediately if hook tab is worn or visibly soiled. Worn tab may cause decannulation.

Check regularly to prevent disengagement.

Discontinue immediately if skin irritation occurs.

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