# NeoGlo<sup>®</sup> Disposable Transilluminator

**REF** Catalog No.

N5536

Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway  
Valencia, CA 91355, USA  
1-800-966-0500  
neotechproducts.com

D514 Rev A

## English

NeoGlo® Disposable Transilluminator

### INDICATIONS FOR USE

The NeoGlo Transilluminator is a hand-held light source intended to transmit light through tissues to aid examination of patients.

### DIRECTIONS FOR USE

Step 1 Remove NeoGlo from packaging. Insert into NeoGlo Sleeve if using.

Step 2 Plug USB cord into laptop, portable power bank (not included), or other USB power source.

Note: While plugging in, ensure that lights are directed away from all persons' eyes.

Step 3 Hold NeoGlo against skin or beneath site to be examined.

Note: Dim room lights for most effective illumination.

Caution: Avoid continuous use over 10 minutes.

Step 4 Unplug when not in use.

Step 5 Clean NeoGlo with alcohol wipe or CaviWipe™ after each use.

Step 6 Recycle or dispose of NeoGlo in accordance with local and federal regulations.

### CAUTIONS & WARNINGS

Prolonged use may cause the surface of the NeoGlo to increase in temperature. Avoid continuous use of more than 10 minutes.

Do not allow direct skin contact with the NeoGlo unless the patient is under the direct supervision of a health care professional.

Use with extreme caution on premature infants, fragile and compromised skin.

The USB cord can pose a potential strangulation hazard.

Do not shine light directly into eyes.

Do not submerge NeoGlo in liquid.

Do not autoclave NeoGlo. Exposure to steam or excessive humidity may cause electrical failure.

Dispose of NeoGlo in accordance with local electronic regulations of electrical products.

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