


NeoShades[®]

PHOTOTHERAPY EYE SHIELDS

N720PL

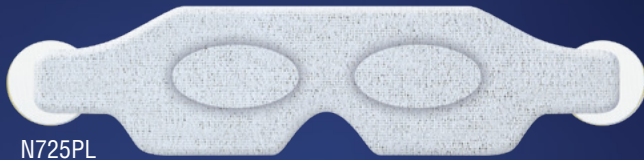

N725PL



N729PL



FEATURES & BENEFITS

- Available with skin friendly NeoBond[®] Hydrocolloid tabs or soft, patented NeoFoam[®] Headstrap to hold NeoShades in place
- Soft, padded eye shield material
- Some SKUs feature ocular pockets designed for patient comfort and safety
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

CAT NO.	ITEM	QTY/UNIT
N720PL	Micro	Tabs 50/box
N720PL-H	Micro	Headstrap 50/box
N721PL	Small	Tabs 50/box
N721PL-H	Small	Headstrap 50/box
N722PL	Large	Tabs 50/box
N722PL-H	Large	Headstrap 50/box
With Ocular Pockets		
N724PL	Micro	Tabs 50/box
N725PL	Small	Tabs 50/box
N726PL	Large	Tabs 50/box
N727PL	Micro	Headstrap 50/box
N728PL	Small	Headstrap 50/box
N729PL	Large	Headstrap 50/box

U.S. Patent #8,070,288;
Additional Patents Pending



NeoShades®

PHOTOTHERAPY EYE SHIELDS

INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Preparation

With Tabs

- 1 Select proper size.
- 2 Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

DO NOT USE ALCOHOL.



Application

- 1 Before removing liners, carefully remove tabs from NeoShades and warm tabs in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.
- 2 Peel and discard liners. Place tabs on skin and hold in place for 60 seconds to ensure proper adhesion.
- 3 Cover eyes with NeoShades and gently attach to tabs.


Note: Leave a portion of the tab exposed for easier removal.



Removal

- 1 Replace NeoShades tabs every 5 days or per hospital protocol, whichever is sooner.
- 2 Hold edge of tab in place and slowly separate NeoShades from tab.
- 3 Saturate tab with water or saline.
- 4 Slowly peel tab away from skin as you swab with water or saline.

Repeat for opposite side.

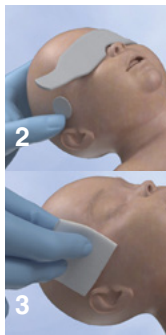

With Headstrap

- 1 Select proper size.
 - 2 Place headstrap behind head with smooth side of foam against head.
 - 3 Cover eyes with NeoShades.
 - 4 Gently attach hook fabric on end of headstrap to NeoShades.
- Repeat for opposite side.

Note: Straps can be trimmed for proper fit.



— CONTRAINDICATIONS, CAUTIONS & WARNINGS —

The NeoShades with Tabs are contraindicated for patients with facial deformation, skin infection or excoriated skin areas that would prevent the proper placement of the tabs.

Tabs should always be removed when NeoShades are taken off the patient. Do not reuse.

The NeoShades are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Discontinue immediately if skin irritation occurs.

Single patient use only.



For more information or to order your **FREE** sample call

1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355



EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague
The Netherlands



PPE Regulation (EU) 2016/425
Notified Body: BSI Group (2797),
Say Building, John M. Keynesplein 9
1066 EP Amsterdam, Netherlands