<p>NEOTECH[®] Products LLC</p>	
Part #	D517
Revision	B
Description	NeoSmile Hydrogel DFU
Artwork #	AW4517
Material # (PM, DM, LM) & description	DM466
Effective date	
CO #	

 Catalog No. N7321

EMERGO EUROPE
Westervoortsejdkj 60
6827 AT, Arnhem, The Netherlands

Medical Device

English
NeoSmile™ Hydrogel Temperature Probe Cover
INDICATIONS FOR USE
The NeoSmile is intended to cover and secure a temperature probe.
DIRECTIONS FOR USE
Preparation
Clean and dry skin per hospital protocol.
Note: It is important for skin to be as clean and dry as possible. NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.
DO NOT USE ALCOHOL.
Application
Step 1 Peel and discard liner. Place NeoSmile Hydrogel over probe and skin.
Step 2 Reposition NeoSmile Hydrogel as needed.
Note: If adhesive is no longer tacky add a drop of water (not saline) to hydrogel and reapply.
Removal
Step 1 Replace NeoSmile Hydrogel every 3 days or per hospital protocol, whichever is sooner.
Step 2 Gently lift edge of NeoSmile and peel back slowly. Water or saline may be applied to the edge to loosen the adhesive bond.
CAUTIONS & WARNINGS
Discontinue immediately if skin irritation occurs.
NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.
Do not use alcohol to clean skin prior to application.
Single patient use only.
European Union: MDR EU 2017/745
Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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