Neotech Bridge® Breastfeeding Assistance Device

| | |
|--|---|
| REF | Catalog No. <p>N8001</p> |
| <p>Manufactured by</p> NEOTECH PRODUCTS LLC <p>28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com</p> | |
| EMERGO EUROPE | |
| EC REF | Westervoortseidijk 60 <p>6827 AT, Arnhem, The Netherlands</p> |
| MD | Medical Device |
| CE | |
| | D518 Rev C 2024.10.01 |

English
Neotech Bridge® Breastfeeding Assistance Device
INDICATIONS FOR USE
The Neotech Bridge is a breastfeeding assistance device intended to deliver supplemental nutrition to an infant latched at the breast/ chest.

DIRECTIONS FOR USE

Step 1 Fill syringe (sold separately for N8001-5 and N8001-30) with milk or formula and twist onto the purple Neotech Bridge connector.
Step 2 Invert the Bridge and center over the nipple. Fold Bridge down onto the breast to create a seal.
Step 3 Latch the baby onto the Bridge.

Step 4 Depress syringe as to deliver milk as needed.

Step 5 Clean after each use and sanitize daily, as directed. Replace Bridge after one week or 20 washes, whichever is sooner.

Cleaning

Clean immediately after each use.

Step 1 Separate and rinse all parts, including syringe plunger, under running water to wash away milk residue.

Step 2 Soak all separate parts in warm soapy water for 5 minutes.
Step 3 Reinsert the plunger into the syringe and fill with warm soapy water. Connect to Bridge and extension tubing (if used; not included for N8001-5 and N8001-30). Flush and repeat several times.

Step 4 Fill syringe with clean water and reconnect to Bridge and extension tubing (if used). Flush and repeat several times.

Step 5 Fill syringe with air and reconnect to Bridge and extension tubing (if used). Force air through tubing several times to remove water droplets.

Step 6 Disconnect all parts, including syringe plunger, and place on a clean, dry surface and allow to air dry. Store dry parts in plastic bag/container when not in use.

Caution: Do not dry the components with paper towel or similar as this will shorten the life of the device. Do not store wet or damp parts.

Sanitizing
Sanitize once daily. Do not use sterilizers or microwave.

Step 1 Separate and rinse all parts, including syringe plunger, under running water to wash away milk residue.

Step 2 Fill a pot with 8 cups of drinking-quality water and bring to a boil.
Step 3 Turn off heat source and lid boiling water cool for no less than two and a half minutes.
Step 4 Place the separate parts into the water and allow them to soak for 10 minutes.

Step 5 Allow water to cool in open pot and gently remove parts from water with tongs.

Step 6 Place parts on a clean, dry surface and allow to air dry. Store dry parts in plastic bag/container when not in use.

Caution: Do not dry the components with paper towel or similar as this will shorten the life of the device. Do not store wet or damp parts.

CONTRAINDICATIONS, CAUTIONS & WARNINGS
The Bridge should not be used if there is an existing medical contraindication for breastfeeding. If unsure, check with your physician.

Do not allow supplement contents to dry in the syringe before washing.

Separate and wash all parts that come in contact with the breast, breastmilk, or supplement immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

Only use drinking-quality tap or bottled water for cleaning. Do not scrub components of the syringe.

Do not expose to extreme heat.

Do not dry the components with paper towel or similar. Do not store wet or damp parts.

Follow cleaning instructions provided by Neotech Products.
European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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