NeoHeart® ECG Pad

REF Catalog No.

N335
N336

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway , Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

MD Medical Device

Made in USA

D509 Rev D

English

NeoHeart® ECG Pad

INDICATIONS FOR USE

NeoHeart is a set of three attached ECG electrodes used on neonates, infants, and pediatric patients.

DIRECTIONS FOR USE

Application

Step 1 Wipe and remove any excess moisture from where NeoHeart will be applied.

Step 2 Peel and discard liner. Apply immediately after liner is removed.

Step 3 Place NeoHeart on patient's back and connect to ECG monitor cable. The pad should be oriented so the point of the heart is pointing toward lower extremities.

Step 4 Confirm quality of ECG tracing.

Note: if QRS is stable but appears small or dampened, consider changing to a different lead on the monitor or increasing the size of the waveform. If small or dampened waveform persists, application to the patient's chest may result in a stronger waveform.

CAUTION: If placed on patient's chest, the white wire electrode will be on patient's left side. Insert into monitor accordingly.

Removal

Step 1 Remove NeoHeart within 24 hours.

Step 2 Saturate the exposed (fabric) side of the NeoHeart with water or saline.

Step 3 Slowly peel NeoHeart away from skin as you continuously swab the skin and adhesive side of the NeoHeart with water or saline.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

NeoHeart ECG Pad should only be used while patient is under continuous direct supervision of healthcare professionals.

NeoHeart is designed for back application. If placed on patient's chest, the white wire electrode will be on patient's left side. Insert into monitor accordingly.

Do not apply to open wounds or broken skin.

Discontinue immediately if skin irritation occurs.

Discard NeoHeart 24 hours after opening pouch.

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