## NeoSnug Tubing Holder

REF Catalog No.

N7015

Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

MD Medical Device

Made in USA

D524 Rev A

English

NeoSnug Tubing Holder

INDICATIONS FOR USE

NeoSnug is intended to support, secure, and hold gastrointestinal

tubes on patients' skin.

DIRECTIONS FOR USE

Preparation

Clean and dry skin where NeoSnug will be placed per hospital

protocol.

Note: It is important for skin to be as clean and dry as possible.

NeoSnug will not adhere properly to moist skin or hair. Oils and

lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing from plastic liner, hydrocolloid adhesive

must be actively warmed. Warming techniques include: hold

between the palms of your hands for at least 60 seconds, hold near a

radiant warmer for 10-15 seconds, or hold against a chemical

heel warmer for 10-15 seconds.

Step 2 One at a time, peel NeoSnug from the clear, printed liner,

place on skin, and hold in place for 60 seconds to ensure proper

adhesion.

Note: If only using one NeoSnug, save second one for later use.

Discard liner when done.

Step 3 Fold down the flap and peel away the white, paper liner

labeled "1" from the base of the NeoSnug. This will expose the

adhesive surface.

Step 4 Lay tubing across the base.

Step 5 Peel away the white, paper liner labeled "2" from the flap and

fold the flap over the tubing to secure.

Note: Once closed, the NeoSnug flap is engineered to remain

permanently in place. You cannot open and close repeatedly.

Removal

Step 1 Replace NeoSnug and attached tubing every 5 days or per

hospital protocol, whichever is sooner.

Step 2 With the tubing still secured, saturate NeoSnug with water

or saline.

Step 3 Slowly peel the NeoSnug away from the skin as you swab

the skin with water or saline.

Step 4 Remove tubing per hospital protocol and discard NeoSnug

and tubing.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a

physician.

Choking hazard. NeoSnug should only be used while the patient is

under the continuous, direct supervision of trained healthcare

professionals.

Risk of ingestion or aspiration.

Discontinue immediately if skin irritation occurs.

NeoSnug will not adhere properly to moist skin or hair. Oils and

lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

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