ENFit[®] Reusable Enteral Syringe

REF Catalog No.

N8020
N8060

Distributed by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway , Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

Medical Device

D521 Rev A

English
ENFit[®] Reusable Enteral Syringe

INTENDED USE

The ENFit[®] Reusable Enteral Syringe is indicated for use as a dispenser, a measuring device and a fluid transfer device. It is used to deliver fluids into the body orally or enterally. It is intended to be used multiple times in non-clinical settings by users ranging from clinicians to laypersons in all age groups. The device is indicated for single patient use only.

INTENDED USER GROUPS AND ENVIRONMENT

Neotech ENFit[®] Reusable Enteral Syringe is intended to be used multiple times in non-clinical settings by users ranging from clinicians to laypersons in all age groups. The device is indicated for single patient use only.

Age groups are as below:


- Neonates/Newborns (birth to 28 days)
- Infants (29 days to less than 2 years old)
- Children (2 years old to less than 12 years old)
- Adolescents (12 years old to less than 18 years old)
- Transitional Adolescents A (18 years old to less than 22 years old)
- Transitional Adolescents B (18 years old to less than 22 years old but treated like adult)
- Adults only (at least 22 years old)

DIRECTIONS FOR USE


Always follow the protocol for using Enteral syringes as advised by the medical healthcare professional.

Step 1 Peel off the package and take out the syringe.

Step 2 Draw nutrient solution or medicine into the syringe. Before drawing liquid such as nutrient solution or medicine liquid into the syringe, push the plunger to the end of the syringe. For accurate measurement use the leading edge of the piston (fig 1). Do not fill beyond the maximum graduation (fig 2).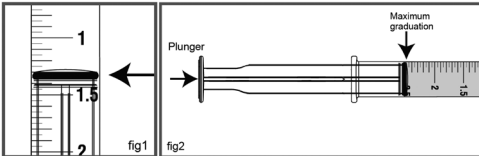

Step 3 Expel air bubbles. To expel air bubbles, hold syringe with tip pointing up. Tap side of syringe to allow bubbles to rise toward tip (fig 3). Draw back slightly on plunger, then push plunger upwards to eject air into the barrel and tip (fig 4). Repeat until all bubbles are eliminated.

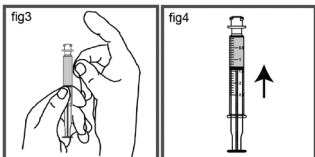Step 4 ENFit syringe connect to gastrointestinal delivery system such as gastric or nasogastric devices to deliver nutritional formulations or medicine to patients physically unable to eat or swallow for various physiological conditions.
Step 5 Use immediately or cap off and label the syringe until ready to use.

CONTRAINDICATIONS, CAUTIONS & WARNINGS

The syringe is not intended for parenteral use.

The syringes should be thoroughly cleaned after use. Dry it ready for the next use.

Stop the use of this reusable syringe past 7 days from its first use or 20 washes (whichever is sooner).

The syringe is intended for reuse on a single patient only.

Do not use if the syringe tip is visibly blocked, or the syringe is damaged.

If the syringe is used with a pump system, follow instructions provided with the pump system. Improper selection of syringe profile in a pump menu may lead to inaccurate delivery to the patient.

Do not connect the device with the following equipment: Intravascular devices; hypodermic applications; breathing systems and driving gas devices; urethral/ urinary devices; limb cuff inflation devices; and neuraxial devices.

Do not use soapy water that contains ester, strong acid or strong alkali to clean the syringe as this will impact the life of the syringe. This device is intended for enteral applications only. Do not use this device with small bore connectors from other healthcare applications as it has the potential risk of misconnection.

Always follow the protocol for using Enteral syringes as advised by the medical healthcare professional.

Syringe must be connected to a male ENFit compatible connector during filling, delivery and any other plunger operation to reduce the risk of dosing volume inaccuracies. When using the syringe with transitional connectors but fail to draw liquid, users should check if the connector is misconnected to the IV set. The connector must be connected with the enteral port, otherwise, the syringe will be unavailable to draw liquid or be used. Neotech does not recommend the use of the "cuffill" technique for filling syringes.

CLEANING INSTRUCTIONS

Clean the syringe immediately after use.

Pull the plunger to the end of the barrel and gently manipulate the plunger from side to side while pulling gently the plunger will come out of the barrel. Rinse the barrel and the plunger in cold tap water. Place on a dry container or clean fresh kitchen roll or paper towel to dry.

To reassemble for the next use:

- Inspect the barrel and plunger ensuring that there are no traces of feed or medicines.
- Check that the inside of the barrel is free from scratches or abrasions and the surface of the piston is not damaged or distorted
- The piston must lie flat and secure in the groove on the plunger.
- Gently manipulate the plunger back into the barrel and push the plunger until the piston covers the zero line of the graduation printed on the barrel.

Soak the syringe in normal dishwashing liquid (40–50° C) for 10 min.

Flush the syringe with cold tap water to flush the residue of dishwashing liquid. According to Point 2 to 4 mentioned in DIRECTIONS FOR USE, use the syringe for a maximum of 7 days or 20 washes before it is discarded and a fresh one used.

The cleaning must be completed. To inspect the device for visible soil following cleaning and to repeat the steps should any be detected.

If the syringe cannot be cleaned effectively or is damaged, discard it and use a fresh one.

The cleaning and rinse water can drain away as normal. There is no need for any special precautions.

Do not attempt to use the syringe if the markings are unclear or the syringe is stiff or difficult to operate.

DISPOSAL

Dispose of syringe as required by Hospital Protocols. If used in home, dispose in household rubbish, or contact the local state healthcare agency.

STORAGE

To be protected from moisture and heat. Should be kept indoors, without corrosive gas, dry, well ventilated and clean.

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