

## SoftSil™

### SILICONE ADHESIVE PATCH



#### FEATURES & BENEFITS

- Great for use as a skin barrier or guard
- Gentle silicone adhesive is ideal for neonatal skin
- Reposition as needed
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	ITEM	QTY/UNIT
N745	Silicone Adhesive Patch	100/box



## SILICONE ADHESIVE PATCH

### INDICATIONS FOR USE

SoftSil Silicon Adhesive Patch intended for general use as a medical adhesive.

### DIRECTIONS FOR USE

- 1 Clean and dry skin per hospital protocol.

*Note: It is important for skin to be as clean and dry as possible. SoftSil will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.*

DO NOT USE ALCOHOL.

- 2 Remove from liner and apply to skin. Reposition as needed.

*Note: Can be cut to size.*

- 3 Replace SoftSil every 2 days or per hospital protocol, whichever is sooner.

Replace immediately if visibly soiled.

### GENERAL INFORMATION

#### Product Materials

Silicone adhesive, medical grade non-woven fabric  
This product is not made with natural rubber latex or plasticizer DEHP.

#### Size

70 mm x 51 mm

#### Packaging

Standard box, no dispense feature

*Visit [neotechproducts.com](https://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly ink on recyclable material.*

### CAUTIONS & WARNINGS

Do not use to secure life-sustaining devices.

Discontinue immediately if skin irritation occurs.

SoftSil will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit **[neotechproducts.com](https://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

