

Neotech Ninni[™]

BREAST-LIKE PACIFIER



FEATURES & BENEFITS

- Flexible breast-like shape supports breastfeeding latch and sucking patterns
- Encourages natural sucking and tongue movement during soothing
- Translucent design for visibility
- Ultra-soft, skin-like texture designed for easy gripping
- Ideal for newborns 0-6 months
- Easy to clean and dishwasher safe
- Can be sterilized in steam sterilization bag
- 100% medical grade silicone
- BPA, PVC, and phthalate free
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

The Neotech Ninni is recommended by Lactation Consultants, Dentists, and Speech Language Pathologists.

CAT. #	SIZE	QTY/UNIT
N8701-25	Standard	25/box

U.S. Patent #10,285,912



Neotech Ninni™

BREAST-LIKE PACIFIER

INTENDED USE

The Neotech Ninni is intended to soothe and support non-nutritive sucking for newborns 0-6 months.

DIRECTIONS FOR USE

Preparation

- 1 Remove Neotech Ninni from packaging.
- 2 Wash thoroughly with warm water and mild soap.
Note: Do not use a brush or other abrasives tools as they may degrade the material.
- 3 Sterilize using an approved steam sterilization bag.
Note: Follow the bag manufacturer's instructions.
Repeat steps 2 and 3 after each use.
Store in a clean, dry container. Do not expose to direct sunlight for prolonged periods.

Inspection

- 1 Visually and manually inspect the Ninni before each use.
- 2 Pull the bulb portion of the nipple in all directions.
- 3 Discard if any of the following are observed: cracks, tears, stickiness, discoloration, thinning of material, or changes in texture

Patient Use

- 1 Place Ninni in infant's mouth and ensure that lips flange around base.
- 2 Place finger in the nipple to help guide the tongue.
Note: if infant doesn't accept the Ninni immediately, reinsert finger and gently press upward against the palate to encourage a latch-like sucking response.

GENERAL INFORMATION

Product Materials

Outer casing: Medical grade silicone
Inner ring: Medical grade silicone
BPA/Phthalate/PVC Free
This product is not made with natural rubber latex or plasticizer DEHP.

Size

Base diameter: 56 mm (2.2 in.)
Nipple base diameter: 32.5 mm (1.28 in.)
Nipple length: 25.5 mm (1 in.)

Packaging

Packaging
Individually packaged
Package type: Polyethylene bag

Visit neotechproducts.com for the most up-to-date information.

CAUTIONS & WARNINGS

Always use the Ninni with adult supervision.

Do not tie Ninni around baby's neck as it presents a strangulation hazard. Only use dedicated pacifier holders tested to EN 12586.

Not intended for chewing; do not use for teething babies.

Inspect the Ninni before each use. Discard immediately if damaged (cracks, tears, stickiness, discoloration, thinning of material, or changes in texture).

Single patient, multi-use.



For more information or to order your FREE sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

