Cannu-Hold™ Tubing and Cannula Holder

REF Catalog No.
N20735

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway , Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

Medical Device

Made in USA

D527 Rev A 2026.07.31

English

Cannu-Hold™ Tubing and Cannula Holder

INDICATIONS FOR USE

The Cannu-Hold is intended to anchor and support oxygen nasal cannula tubing on the intact skin of pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Preparation

Clean and dry skin where Cannu-Hold will be placed per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Cannu-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing from plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10- 15 seconds, or hold against a chemical heat warmer for 10-15 seconds.

Step 2 One at a time, peel Cannu-Hold from the clear, printed liner, place on skin, and hold in place for 60 seconds to ensure proper adhesion.

Step 3 Fold down the flap and lay tubing across the base.

Step 4 Peel away the clear plastic liner to expose the adhesive surface and fold flap over tubing.

Note: Lift flap as needed to reposition or remove tubing.

Removal

Step 1 Replace Cannu-Hold every 5 days or per hospital protocol, whichever is sooner.

Step 2 Lift flap to remove tubing.

Step 3 Saturate Cannu-Hold with water or saline.

Step 4 Slowly peel Cannu-Hold away from skin as you swab the skin with water or saline.

CAUTIONS & WARNINGS

Choking hazard. Cannu-Hold should only be used while the patient is under the continuous, direct supervision of a caregiver.

Risk of ingestion or aspiration.

Discontinue immediately if skin irritation occurs.

Cannu-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

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