


Cannu-Hold

TUBING AND CANNULA HOLDER

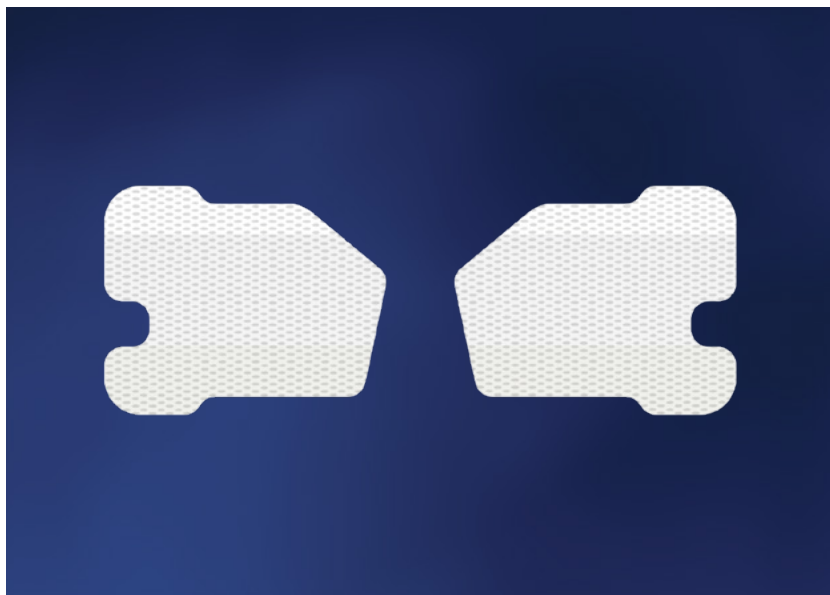

FEATURES & BENEFITS

- Skin friendly NeoBond[®] Hydrocolloid adhesive base
- Unique shape ideal for securing tubes close to the nares
- Fold over flap engineered to relieve stress and keep tubes securely in place
- Flap designed to be opened and closed repeatedly
- Non-woven fabric contours to face or body
- Two per set with left and right tube holders
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	SIZE	QTY/UNIT
N7035	Small	70/box

*35 sets of 2 (left/right)

Patent Pending



Cannu-Hold

TUBING AND CANNULA HOLDER

INDICATIONS FOR USE

The Cannu-Hold is intended to anchor and support oxygen nasal cannula tubing on the intact skin of pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Preparation

Clean and dry skin where Cannu-Hold will be placed per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Cannu-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

- 1 Before removing from plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10- 15 seconds, or hold against a chemical heel warmer for 10-15 seconds.
- 2 One at a time, peel Cannu-Hold from the clear, printed liner, place on skin, and hold in place for 60 seconds to ensure proper adhesion.
- 3 Fold down the flap and lay tubing across the base.
- 4 Peel away the clear plastic liner to expose the adhesive surface and fold flap over tubings.

Note: Lift flap as needed to reposition or remove tubing.

Removal

- 1 Replace Cannu-Hold every 5 days or per hospital protocol, whichever is sooner.
- 2 Lift flap to remove tubing.
- 3 Saturate Cannu-Hold with water or saline.
- 4 Slowly peel Cannu-Hold away from skin as you swab the skin with water or saline.

GENERAL INFORMATION

Product Materials

NeoBond Hydrocolloid, non-woven fabric, transfer tape (does not contact skin), repositionable differential tape (does not contact skin)

This product is not made with natural rubber latex or plasticizer DEHP.

Size

N7035 Small closed: 40 mm x 32 mm

Visit neotechproducts.com for the most up-to-date information.

CAUTIONS & WARNINGS

Choking hazard. Cannu-Hold should only be used while the patient is under the continuous, direct supervision of a caregiver.

Risk of ingestion or aspiration.

Discontinue immediately if skin irritation occurs.

Cannu-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Disposable. Do not reuse.

Single patient use only.



For more information or to order your **FREE** sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

